



Pasweg 6A
B-3740 Bilzen


Tel.: (+32) (0) 89/510.510
Fax: (+32) (0) 89/510.511

www.gymna-uniphy.com
E-mail: info@gymna-uniphy.com

Your dealer:



User Manual

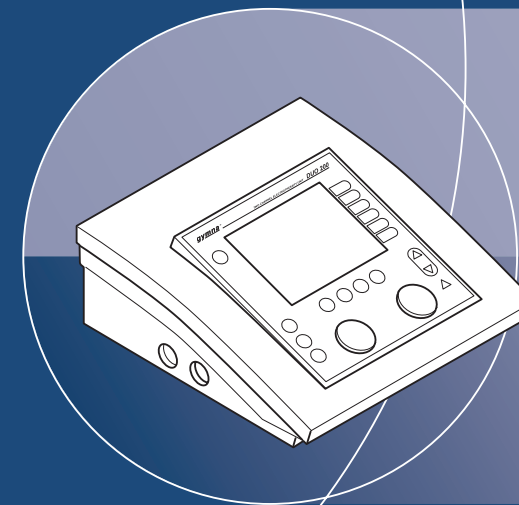

DUO 200

User Manual Duo 200

Device for electrotherapy

Manufacturer	GymnaUniphy N.V.
Main office	Pasweg 6A B-3740 BILZEN
Telephone	+(32) (0)89-510.510
Fax	+(32) (0)89-510.511
E-mail	info@gymna-uniphy.com
Website	www.gymna-uniphy.com

CE 0344

Version 1.1

March 2005

Abbreviations

AQ	Accommodation Quotient
CC	Constant Current
CP	Courte Période
CV	Constant Voltage
DF	Diphasé Fixe
EL	Electrode
EMC	Electromagnetic Compatibility
ESD	Electrostatic Discharge
ET	Electrotherapy
HAC	Hospital Antiseptic Concentrate
LP	Longue Période
MF	Medium Frequency: with unidirectional and interferential currents Monophasé Fixe: with diadynamic currents
MTP	Myofascial Trigger Point
NMES	Neuro Muscular Electro Stimulation
TENS	Transcutaneous Electrical Nerve Stimulation
VAS	Visual Analogue Scale

Symbols on the equipment



Read the manual



Manufacturer

Symbols in the manual



Warning or important information.

TABLE OF CONTENTS

1	SAFETY	7
1.1	PURPOSE	7
1.2	SAFETY INSTRUCTIONS	7
1.3	MEDICAL DEVICES DIRECTIVE	9
1.4	LIABILITY	9
2	INSTALLATION	11
2.1	RECEIPT	11
2.2	PLACING AND CONNECTION	11
2.3	PERFORMING THE FUNCTIONAL TEST	11
2.4	SETTING CONTRAST AND SELECTING LANGUAGE	11
2.5	TRANSPORT AND STORAGE	12
2.6	RESELLING	12
3	DESCRIPTION OF THE EQUIPMENT	13
3.1	DUO 200 AND STANDARD ACCESSORIES	13
3.2	COMPONENTS OF DUO 200	14
3.3	DISPLAY	15
3.4	DISPLAY SYMBOLS	16
3.5	SYMBOLS FOR CURRENT SHAPES IN MEMORY MENU	16
3.6	PARAMETER SYMBOLS	17
3.7	CURRENT SHAPES	17
4	OPERATION	19
4.1	THERAPY SELECTION	19
4.2	PERFORMING THERAPY	21
4.3	ELECTROTHERAPY	23
4.4	DIAGNOSTIC PROGRAMS	31
4.5	PROGRAMS	33
4.6	SYSTEM SETTINGS	35
5	INSPECTIONS AND MAINTENANCE	39
5.1	INSPECTIONS	39
5.2	MAINTENANCE	40
6	MALFUNCTIONS, SERVICE AND GUARANTEE	43
6.1	MALFUNCTIONS	43
6.2	SERVICE	44
6.3	GUARANTEE	44
6.4	TECHNICAL LIFE TIME	45

7	TECHNICAL INFORMATION	47
	7.1 GENERAL	47
	7.2 ELECTROTHERAPY	47
	7.3 ENVIRONMENTAL CONDITIONS	49
	7.4 TRANSPORT AND STORAGE	50
	7.5 STANDARD ACCESSORIES	50
	7.6 OPTIONAL ACCESSORIES ELECTROTHERAPY	51
8	APPENDICES	53
	8.1 AGENTS FOR IONTOPHORESIS	53
	8.2 DIAGNOSTIC I/T-CURVE	54
	8.3 EMC DIRECTIVE	55
	8.4 TECHNICAL SAFETY INSPECTION	59
	8.5 DISPOSAL	62
9	REFERENCE	63
	9.1 FUNCTION OVERVIEW	63
	9.2 LITERATURE	67
	9.3 TERMINOLOGY	67
10	INDEX	71

1 SAFETY

1.1 Purpose

The Duo 200 is intended solely for medical applications. You can use the Duo 200 for electrotherapy. The device is suited for continuous use.

1.2 Safety instructions

1.2.1 General



- Only qualified people who are trained in the application of the therapies may use the appliance.
- Only a technician authorised by GymnaUniphy N.V. may open the equipment or the accessories.
- Follow the instructions and directions in these user instructions.
- Place the equipment on a horizontal and stable base.
- Keep the ventilation openings at the bottom and rear of the equipment free.
- Do not place any objects on the equipment.
- Do not place the equipment in the sun or above a heat source.
- Do not use the equipment in a damp area.
- Do not let any liquid flow into the equipment.
- Do not disinfect or sterilise the equipment. Clean the equipment with a dry or moistened cloth. *See §5.*
- Only treat patients with electrical implants (pacemaker) after obtaining medical advice.
- The 'Directive on Medical Devices' from the European Commission (93/42/EEG) requires that safe devices are used. It is recommended to perform a yearly technical safety inspection. *See §5.1.1.*
- For optimum treatment, a patient investigation must first be performed. On the basis of the findings of the investigation, a treatment plan with objectives will be formulated. Follow the treatment plan during the therapy. This will limit possible risks, related to the treatment, to a minimum.
- Always keep these user instructions with the equipment.

1.2.2 **Electrical safety**



- Only use the equipment in an area with facilities that meet the applicable legal regulations.
- Connect the equipment to an outlet with a protective earth terminal. The outlet must meet the locally applicable requirements for medical areas.

1.2.3 **Prevention of explosion**



- Do not use the equipment in an area where combustible gases or vapours are present.
- Switch off the equipment when it is not used.

1.2.4 **Electro Magnetic Compatibility**



- Medical electrical equipment requires special precautions for Electro Magnetic Compatibility (EMC). Follow the instructions for the installation of the equipment. *See §2.*
- Do not use mobile telephones or other radio, shortwave, or microwave equipment in the vicinity of the equipment. This kind of equipment can cause disturbances.
- Only use the accompanying accessories that are supplied by GymnaUniphy. *See §7.5 and §7.6.* Other accessories can lead to an increased emission or a reduced immunity.

1.2.5 **Electrotherapy**



- Do not use the equipment simultaneously with high frequency surgical equipment. This combination can cause burning of the skin under the electrodes.
- Do not use adhesive electrodes with currents that have a galvanic component, such as galvanic, diadynamic, MF rectangular, pulsed rectangular and triangular currents. With these currents, etching of the skin can occur.
- Check the electrode cables and the electrodes at least once a month. Check whether the insulation is still intact. *See §5.1.*
- The safety standards for electrical stimulation advise not to exceed the current density of $2.0 \text{ mA}_{\text{rms}}/\text{cm}^2$. However, with iontophoresis treatments, we advise a maximum current density of $0.25 \text{ mA}/\text{cm}^2$, because of using the MF rectangular current. Exceeding this value can result in skin irritation and burns.
- Always use sterilised gauze with iontophoresis treatments.

1.3 **Medical Devices Directive**

The device complies with the essential requirements of the Medical Device Directive of the European Committee (93/42/EEC) as most recently changed.

1.4 **Liability**

The manufacturer cannot be held liable for injury to the therapist, the patient or third parties, or for damage to or by the equipment used, if for example:

- an incorrect diagnosis is made;
- the equipment or the accessories are used incorrectly;
- the user instructions are wrongly interpreted or ignored;
- the equipment is badly maintained;
- maintenance or repairs are performed by people or organisations that are not authorised to do so by GymnaUniphy.

Neither the manufacturer nor the local GymnaUniphy dealer can be held liable, in any way whatsoever, for the transfer of infections via the vaginal, anal and rectal probes and/or other accessories.

2 INSTALLATION

2.1 Receipt

1. Check whether the equipment has been damaged during transport.
2. Check whether the accessories are intact and complete. *See §7.5 and §7.6.*
 - Inform your supplier of any damage or defects by no later than within 3 working days after receipt. Report the damage by telephone, fax, e-mail or letter.
 - Do not use the equipment if it is damaged or defective.


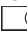
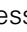
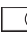




2.2 Placing and connection

1. Place the equipment on a horizontal and stable base.
 - Keep the ventilation openings at the bottom and rear of the equipment free.
 - Do not place the equipment in the sun or above a heat source.
 - Do not use the equipment in a wet area.
2. Check whether the mains voltage that is stated on the rear of the equipment corresponds with the voltage of your mains supply. The equipment is suited for a nominal mains voltage from 100 V to 240 VAC / 50-60 Hz.
3. Connect the device to an outlet with protective earth terminal.

2.3 Performing the functional test

1. Switch the equipment on with the switch at the rear of the equipment.
2. When the equipment is switched on, it automatically performs a test. Check whether the indicator lamps next to **Y_A** and **Y_B** light briefly during the test.
3. If the lamps do not light up: *See §6.*

2.4 Setting contrast and selecting language

1. Press  for 5 seconds. The **System setting** menu appears. *See §4.6.*
2. Press  next to **Contrast**, 1st key from the top.
3. If necessary, change the contrast with  and .
4. Press  next to **Language**.
5. If necessary, change the language with  and .
6. Press  to return to the start menu.

2.5 Transport and storage

Take account of the following matters if the equipment has to be transported or stored:


- Transport or store the equipment in the original packaging.
- The maximum period for transport or storage is: 15 weeks.
- Temperature: -20 °C to +60 °C.
- Relative humidity: 10% to 100%.
- Atmospheric pressure: 200 hPa to 1060 hPa.

2.6 Reselling

This medical equipment must be traceable. The equipment and some other accessories have a unique serial number. Provide the dealer with the name and address of the new owner.


3 DESCRIPTION OF THE EQUIPMENT

3.1 Duo 200 and standard accessories

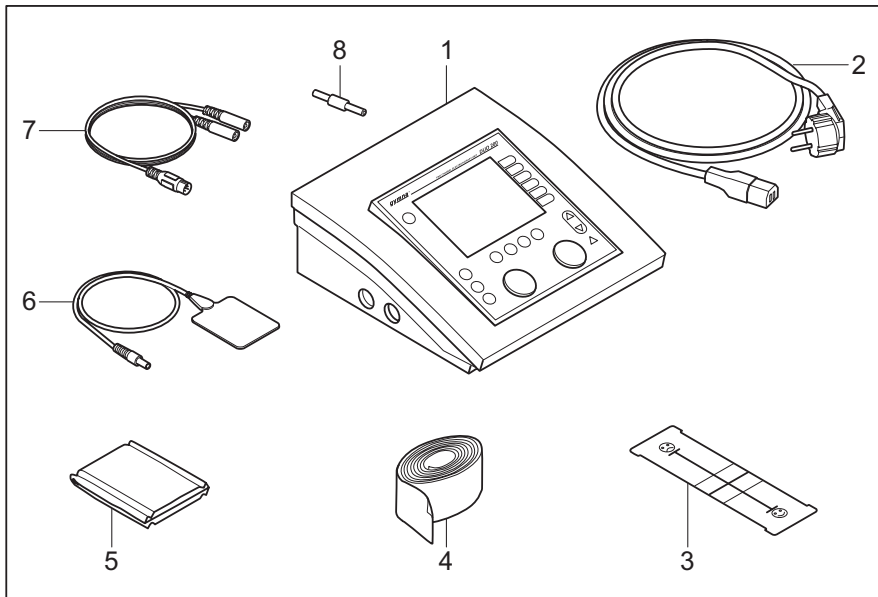


- | | |
|---|---------------------------------------|
| 1. Duo 200. See §3.2. | 6. Rubber electrodes (4 pieces) |
| 2. Power cord | 7. Two-ply electrode cable (2 pieces) |
| 3. VAS score card | 8. Test connector |
| 4. Elastic fixation straps (4 pieces) | |
| 5. EL sponges for rubber electrode (4 pieces) | |

3.2 Components of Duo 200

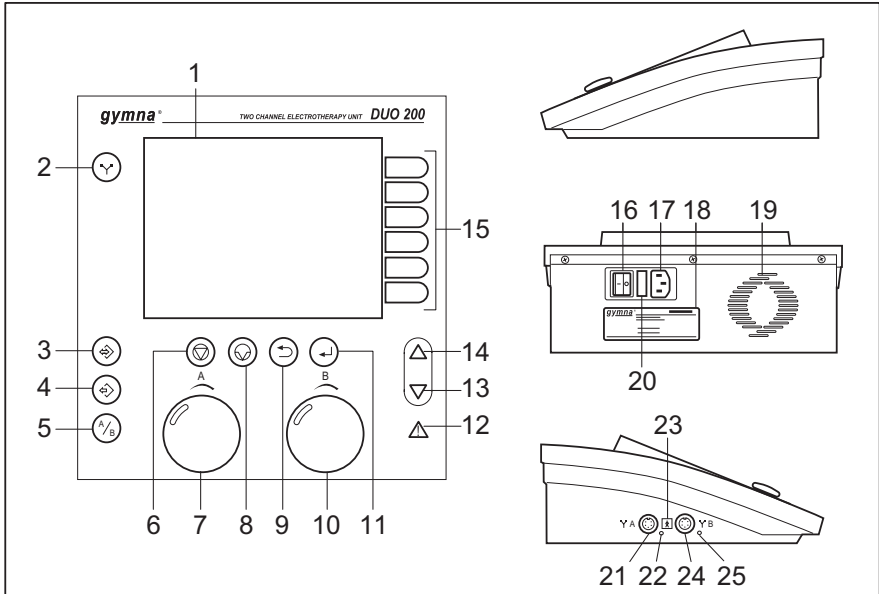

- | | |
|--|---|
| <ol style="list-style-type: none"> 1. Display. See §3.3. 2. Electrotherapy 3. Memory 4. Start menu 5. Channel selection: A or B 6. Stop 7. Intensity of channel A 8. Pause 9. Return to previous menu 10. Intensity of channel B 11. Enter 12. Indication: Read manual 13. Down 14. Up | <ol style="list-style-type: none"> 15. Select parameter or menu 16. On/off switch 17. Connection to mains supply 18. Type plate 19. Ventilation opening 20. Fuse holder 21. Connector for electrotherapy, channel A 22. Indicator lamp for channel A 23. Indication: Floating patient circuit 24. Connector for electrotherapy, channel B 25. Indicator lamp for channel B |
|--|---|

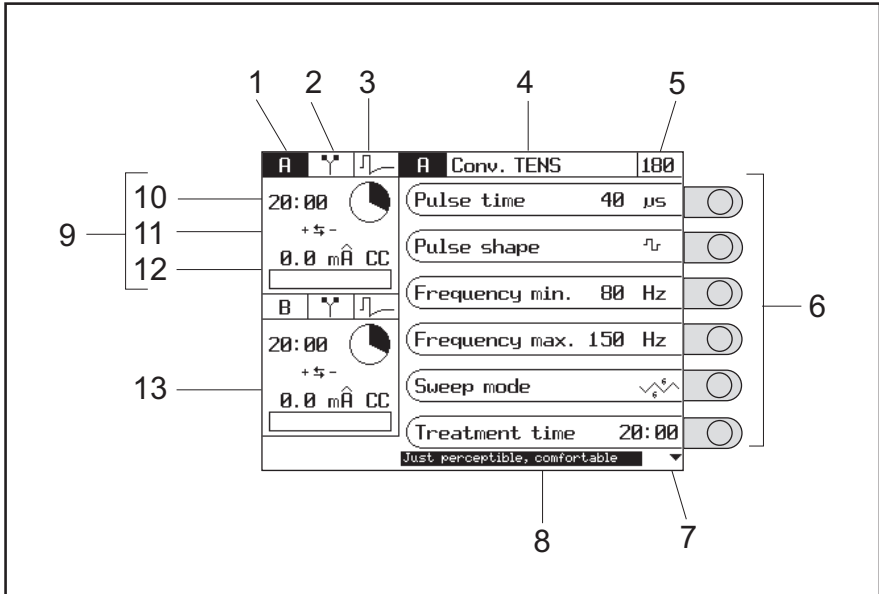


3.3 Display









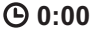

1. Channel
2. Electrotherapy
3. Current shape
4. Title of the screen
5. Program number
6. Parameters with selection knobs
7. Use ∇ to go to the next parameters
8. Explanation or recommendation
9. Screen for channel A. See §4.3.5.
10. Remaining treatment time
11. Polarity
12. Set intensity
13. Screen for channel B

3.4 Display symbols














3.4.1 General




	Electrotherapy	A	Channel A
SEQ	Sequential current shapes	B	Channel B
	Treatment time	A + B	Channel A and B simultaneously
	Treatment completed		

3.4.2 Current shape groups



	Unidirectional currents		2-pole medium frequency
	Diadynamic		4-pole Interferential
	TENS currents		4-pole interferential with vector
	NMES currents		Diagnostic programs

3.5 Symbols for current shapes in memory menu

	Medium frequency unidirectional current		Burst TENS
	Unidirectional rectangular current		Rectangular surge current
	Unidirectional triangular current		Triangular surge current
	Conventional TENS		Biphasic surge current
	Low frequency TENS		Intrapulse interval surge current
	Random TENS		2-pole medium frequency surge current
CP	CP (diadynamic)		2-pole medium frequency

DF	DF (diadynamic)		4-pole interferential with rotating vector
LP	LP (diadynamic)		Rheobase and chronaxie
MF	MF (diadynamic)		Rheobase and AQ

3.6 Parameter symbols





Red+ Red-	Polarity indication	CC	Constant Current
+ ⇌ -	Alternating polarity	CV	Constant Voltage
	Biphasic pulse shape, symmetrical	mÂ	mA peak
	Biphasic pulse shape, asymmetrical	∧	Volt peak

3.6.1 Sweep mode

	12s/12s		1s/5s -1s/5s
	6s/6s		1s/1s

3.7 Current shapes

3.7.1 Unidirectional currents

	Rectangular pulse current		Triangular pulse current
	2-5 current (UltraReiz)		Medium frequency rectangular current

3.7.2 Diadynamic currents

	MF		CP
	DF		LP

3.7.3 Interferential currents



2-pole medium frequency



4-pole interferential with rotating vector



4-pole Interferential

3.7.4 TENS currents




Conventional TENS, asymmetrical




Conventional TENS, alternating symmetrical




Conventional TENS, alternating asymmetrical

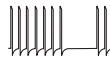

TENS burst

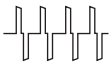


Conventional TENS, symmetrical

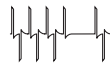


TENS burst, alternating

3.7.5 NMES currents

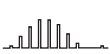

Rectangular surge current

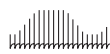

Biphasic surge current



Triangular surge current



Intrapulse interval surge current



Medium frequency surge current (2- and 4-pole)

4 OPERATION

4.1 Therapy selection


You can select a therapy in different ways, with the therapy key or with the parameters in the **Start menu**:

- **Therapy keys:** Quickly select a therapy with therapy keys (⏏). See §4.1.1.
- **Objectives:** Select a therapy on the basis of an objective. See §4.1.2.
- **Indication list:** Select a therapy on the basis of a medical indication. See §4.1.3.
- **Program number:** Select a certain program number or a program number that you previously saved. See §4.1.4.
- **Diagnostic programs:** Perform a diagnosis, for example to determine the rheobase and the chronaxie. See §4.1.5.
- **Contra indications:** Display an overview with contra indications for the electrotherapy. See §4.1.6.

Besides this, you can change the system settings. See §4.6.


4.1.1 Therapy keys

1. Press (⏏): **Electrotherapy**.
2. Select the current shape group with .
3. Select the current shape with .

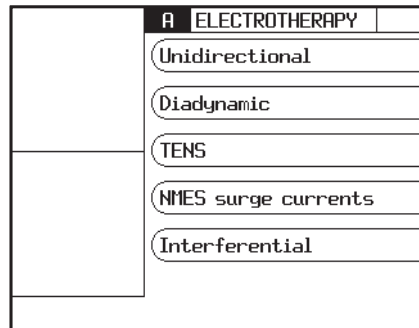


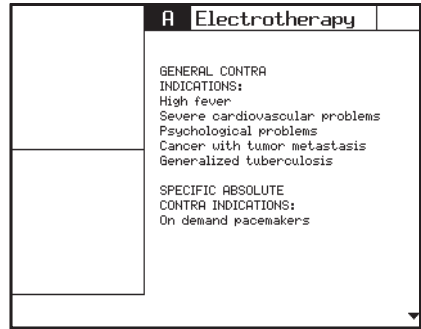
4.1.2 Therapy selection via objectives

1. Press (⏏) to go to the start menu.
2. Select **Objectives**.
3. Select the desired treatment with .

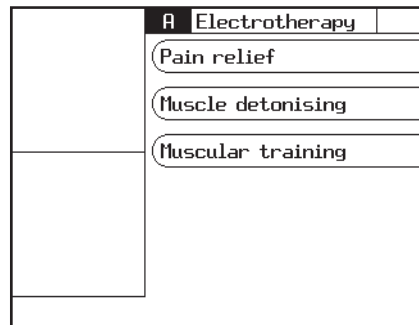








4.1.6 *Contra indication selection*


1. Press  to go to the start menu.
2. Select **Contra indications**.



4.2 Performing therapy

4.2.1 *Set and start therapy*

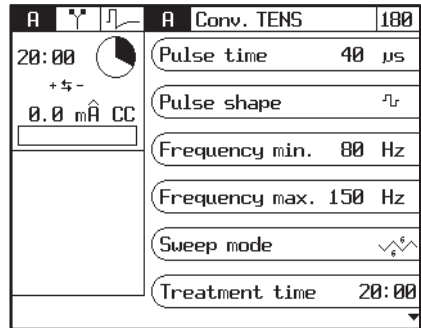


1. Press  to go to the start menu.
2. Select the desired menu with  until the treatment appears.
3. Select the desired parameters with . You can only change the outlined parameters.
4. Set the **Treatment time** as follows: Press once on  to set the minutes, press twice on  to set the seconds.
5. Change the value of the parameter with  and . The setting range of the parameter is shown at the bottom of the screen. You can change the parameter as long as the parameter has a black background.
6. Rotate intensity knob A or B to start the treatment and to set the desired intensity. The set intensity is displayed in the screen.



4.2.2 Set channels A and B

The Duo 200 has two separated electrotherapy channels A and B. The only restriction is that both are in the CC mode or the CV mode.

The channels A and B can be used independently. You can treat two different indications simultaneously with two different treatments.



1. Press  for 5 seconds. The **System setting** menu appears. See §4.6.
2. If necessary, change the parameter **Copy parameters** to **OFF**.
3. The selected channel has a black background. If desired, press  to change the first channel.
4. Select a treatment. See §4.1.
5. Set the parameters for the first channel. See §4.2.1.
6. Press  to select the other channel.
7. Select a treatment for the second channel. See §4.1.
8. Set the parameters for the second channel. See §4.2.1.

Both channels are selected simultaneously and automatically in case of:



- 4-pole current shapes
- Combination therapy

Copy channel


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
On the second channel, you can set the same parameters for electrotherapy as for the first set channel.


1. Press  for 5 seconds. The **System setting** menu appears. See §4.6.
2. If necessary, change the parameter **Copy parameters** to **ON**.
3. Select a treatment. See §4.1.
4. Set the parameters for the first channel. See §4.2.1.
5. Press  to select the other channel. The treatment including the settings are copied to the other channel.
6. If desired you can change the parameters or the treatment of the selected channel.

Clear channel


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

1. Make sure that the intensity is set to zero.
2. Press  to select the channel that you want to clear.
3. Press . The channel is cleared.

4.2.3 Opening the intensity screen


1. Set the treatment. See §4.2.1.
2. Rotate intensity knob A or B to start the treatment.
3. Press . The intensity screen appears.
The left part of the screen shows channel A. The right part of the screen shows channel B.
4. Press  to return to the setting menu.

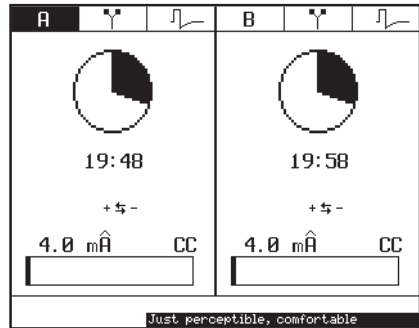

4.2.4 Temporary interruption of treatment

1. If the other channel has to pause: Select this channel with .
2. Press  during the treatment. The treatment time of the selected channel is stopped. **Pause** appears on the screen. The parameter settings are retained.
3. Press on  again to restart the treatment. The intensity now increases gradually to the set level and the treatment time continues again.

4.2.5 Immediately stop treatment

1. Press . All active treatments are stopped immediately. **Stop** appears on the screen. The parameter settings are retained.
2. Set the intensity of the channel again to continue the treatment.


4.3 Electrotherapy

4.3.1 Performing electrotherapy with electrodes

1. Select the desired electrotherapy program.
2. Place the electrodes. See page 24: *Placing rubber electrodes* and page 24: *Placing the adhesive electrodes*. With some treatments, the **Electrode placing** parameter refers to the number in the placing diagrams.
3. Rotate intensity knob A or B to start the electrotherapy and to set the desired intensity. See §4.1.2.
4. Check the patient's reaction. Repeat this check regularly during the treatment.
5. The equipment stops the treatment and indicates that the treatment is completed. Remove the electrodes.

Placing rubber electrodes

1. Moisten two EL sponges. Use water with a saline solution to improve the conductivity of the EL sponges.
2. Slide a rubber electrode into each sponge.
3. Place the sponges on the part of the body that must be treated.
4. Fasten the sponges to the part of the body with the elastic fixation straps.
5. Connect the rubber electrode with the red connector to the red connector of the two-ply electrode cable.
6. Connect the rubber electrode with the black connector to the black connector of the two-ply electrode cable.
7. Connect the two-ply cable to connector **Y**A or **Y**B of the Duo 200.

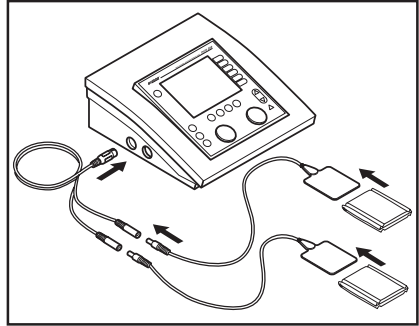


Placing the adhesive electrodes



Do not use adhesive electrodes with currents that have a galvanic component, such as galvanic, diadynamic, MF rectangular, pulsed rectangular and triangular currents. These currents can cause skin etching.

1. If possible, disinfect the parts of the body where the adhesive electrodes are to be placed.
2. Place the electrodes on the part of the body that must be treated.
3. Connect the connectors of the adhesive electrodes to the adapter cables.
4. Connect the adapter cables to the two-ply electrode cable.
5. Connect the two-ply electrode cable to connector **Y**A or **Y**B of the Duo 200.

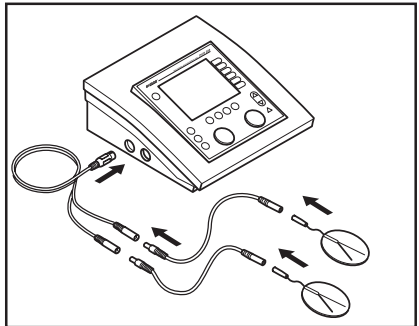


4.3.2 Perform electrotherapy with vaginal, anal or rectal stimulation probe



- Considering the very personal and intimate character of these treatments, a probe may only be used for one patient.
- Never disinfect the probes in an autoclave. The probes can be damaged by the extreme temperature.

1. Clean the probe carefully with soap and water.
2. Select the desired electrotherapy program.
3. Connect the probe to the Duo 200. The vaginal and anal probes are immediately detected by the equipment. To prevent unpleasant stimulations, you can only set alternating currents with a Constant Voltage (CV) setting, such as TENS, NMES, and 2-pole interferential currents.

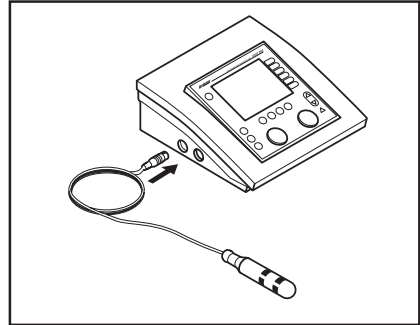

The rectal stimulation probe is not detected by the equipment. With a rectal stimulation probe, select only alternating currents with a Constant Voltage (CV) setting, such as TENS, NMES, and 2-pole interferential currents. This prevents skin etching and unpleasant stimulations.

4. Apply an antiseptic lubricant to the probe.
5. Place the probe.
6. Rotate intensity knob A or B to start the treatment and to set the desired intensity.
7. Check the patient's reaction. Repeat this check regularly during the treatment.
8. The equipment stops the treatment and indicates that the treatment is completed. Remove the stimulation probe.
9. Clean the stimulation probe. See §5.2.4.

4.3.3 Electrotherapy with sequential steps

A treatment with sequential steps consists of a succession of the same current form, but additional with different parameter settings. You can set the time between the steps.

Advantages

Electrotherapy with sequential steps has several advantages:


- In one electrotherapy, you can realise several objectives.
- In a treatment with one objective, you can place different accents in the objective.
- You can distinguish between different phases in a treatment, for example preparation, core effect and cooling.

Set new intensity between sequential steps

The intensity determines the peak value during the treatment. When changing to a following step, the intensity is retained if safety allows. Sometimes, it is necessary to increase the intensity for the following step. If the intensity cannot be maintained for safety reasons, the intensity returns to zero. In this case, the treatment is stopped. You must now set the intensity again.


Setting a treatment with sequential steps

1. Select a treatment whereby you can set sequential steps, for example with **Objectives, Electrotherapy, Muscular training, Specific muscle functions, Endurance**.
You can also select a program directly with the program number.
2. Set the **Step time** and **Stimulation beep** parameters for the different steps.

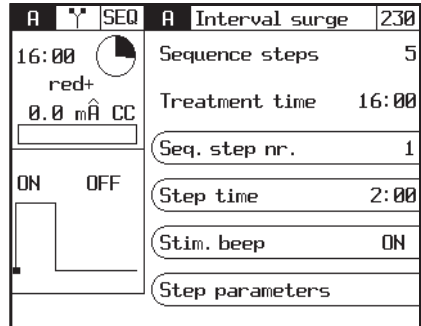

3. Rotate intensity knob A or B to start the treatment and to set the desired intensity.

Skip step in treatment

1. Press  to temporarily interrupt the treatment.
2. Select **Seq. step nr.** and select the desired step.
3. Rotate intensity knob A or B to continue the treatment again and to set the desired intensity.

4.3.4 Performing iontophoresis


1. Apply the medicament on a sterile gauze. See §8.1. Care must be taken in administering medicaments (allergies, contra indications, ...).
2. Place the gauze on the electrode. Make sure that the polarity corresponds with the medicament used.
3. Place the electrodes. See page 24: *Placing rubber electrodes* and page 24: *Placing the adhesive electrodes*.
4. Select **Electrotherapy, Unidirectional, Medium freq. constant**.
5. **Set the intensity between 0.1 and 0.25 mA/cm²**. The intensity depends on the surface area of the electrodes. With electrodes of 6 x 8 cm (=48 cm²), the current setting must be between 4.8 and 12 mA.



To prevent etching or burns, never exceed 0.25 mA/cm².

4.3.5 Read-out values

1. Channel
2. Electrotherapy
3. Current shape
4. Remaining treatment time
5. Polarity
6. Present intensity
7. Graphical representation of intensity
8. Progress of current

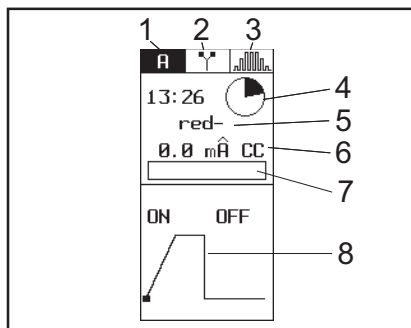

Progress of current

With NMES currents and 4-pole current shapes, the progress of the current is graphically displayed. This gives a clear insight into the phase in which the current is at that moment. In this way, you can optimally guide the patient during the execution of the exercise. With the simultaneous application of two NMES currents, the current is only graphically displayed in the intensity screen.

Press to open the intensity screen.

4.3.6 Parameters

The following parameters are given alphabetically. The setting range or the selection possibilities of the parameters depend on the treatment chosen.

Active rest (s)

The duration of the rest period. During the rest period, a low frequency current is applied to stimulate the recovery process.

Burst (Hz)

The frequency of the biphasic pulses. The burst consists of a series of pulses that is repeated several times per second. Each burst consists of a low frequency current with high internal pulse frequency (70 - 100 Hz) and a long pulse duration (100 - 250 μ sec).

Carrier wave (kHz)

The carrier wave frequency, expressed as the number of cycles per second. The frequency of this medium frequency current corresponds with the cycle duration. A high frequency results in a short pulse duration. A carrier wave frequency of 2 kHz is suited for muscle stimulation.

CC / CV

Constant Current (CC) or Constant Voltage (CV).



- When using a dynamic electrode technique, only use alternating currents with Constant Voltage (CV). This prevents unpleasant stimulations for the patient when the contact is temporarily interrupted during the placement, movement and removal of the electrode.
- With a rectal stimulation probe, select only alternating currents with Constant Voltage (CV), such as TENS, NMES, and 2-pole interferential currents. This prevents skin etching and unpleasant stimulations. The rectal stimulation probe is not detected by the equipment.

Characteristics of Constant Current:

- The voltage increases with an increasing load impedance (a worsening contact).
- Within the stated limits, a variation in the load impedance has hardly any effect on the current.
- Without a load, the voltage will go to the maximum level within a short time. After this, an error message will appear on the screen and the current will be switched off.

Characteristics of Constant Voltage:

- With a decreasing load impedance, the current increases.
- Without a load, the output voltage is equal to the set value.
- With a short circuit, the output current in mA is equal to the set voltage in V.

Electrode placing

Instructions for placing the electrodes. Consult the placement diagrams.

Frequency min./max. (Hz)

The minimum and maximum frequency of the current cycles, expressed as the number of cycles per second. Within the set sweep mode, the frequency changes within these limits. During the treatment, frequency modulation is desired to prevent habituation. It is recommended to select a fairly low minimum frequency for this (< 20%).

Isodynamic (on, off)

LP and CP use two phases: MF and DF. The MF phase is more intense than the DF phase. If the patient is very sensitive, this difference in perception can be adjusted with this parameter.

On: Reduce the amplitude of the MF phase by 12.5%.

Off time (off) (s)

The interval between two series of current pulses.

On2 amplitude

The amplitude of the pulses during the **On2** period. This amplitude can be set as a percentage of the set amplitude during the **On** period.

On2 frequency

The frequency of the pulses during the **On2** time.

On time (on) (s)

The time that the series of current pulses is switched on.

Polarity

The polarity of the current pulse.

Polarity change (on, off)

Switch polarity between red+ and red- during the treatment.

Pulse pause (ms or s)

The duration between the current pulses.

Pulse shape

The shape of the electrical pulse. See §3.7.

Pulse time (µs, ms or s)

The duration of the current pulse.

Rest amplitude (%)

The amplitude of the pulses that is maintained during the active rest period. The active rest period stimulates recovery, which is otherwise realised by the "Off time". The amplitude during the active rest period is set as a percentage of the amplitude during the "On time".

Rest frequency (Hz)

The frequency that is maintained during the active rest period of the NMES current.

Rotation angle (0 - 355°)

The actual angle between the line with the maximum amplitude and the line between the electrodes of channel B. If **Manual** is selected for **Rotation mode**, you can let this angle rotate step by step. This makes it possible to localise deeper treatment points.

Rotation mode (manual, auto)

The maximum amplitude is present at one line in the rotation field (with 100% modulation depth).

- **Auto:** The line with maximum amplitude and 100% modulation depth automatically rotates 360° through the interference field during the set rotation time.
- **Manual:** Position this line manually in the interference field. You do not need to move the electrodes for this.

Rotation time (0 - 20 s)

The time in which the line with maximum amplitude and 100% modulation depth rotates 360° through the interference field. Use a short rotation time (3 - 5 s) to prevent habituation. Use a long rotation time (10-15 s) to localise deeper treatment points.

Segment angle (0, 15, 30°)

With the segment angle, a certain segment can be stimulated. The segment angle can be set when the **Rotation angle** is set to **Manual**.

Segment time (s)

The time in which the rotation angle changes within the set segment angle.

Seq. step nr. (1 - 5)

The number of the sequential step that is activated. See §4.3.3.

Seq. steps

The maximum number of sequential steps. See §4.3.3.

Step time (mm:ss)

The time in which the selected sequential step number is performed.

Stimulation beep (on, off)

Switch stimulation beep on or off.

Sweep mode

This parameter is only available if **Frequency min** deviates from **Frequency max**. The frequency cycle consists of four steps with fixed set values: increase, hold, decrease and hold. During the treatment, frequency modulation is desired to prevent habituation.

Treatment time (mm:ss)

The duration of the treatment.

4.4 Diagnostic programs

With the diagnostic programs, you can investigate the state of the electrical sensitivity of the neuro-muscular system:

- Rheobase and chronaxie. See §4.4.1.
- Rheobase and AQ. See §4.4.2.
- Manually determine an I/T curve. See §4.4.3.

Besides this, there are diagnostic programs for localisation:

- Pain points. See §4.4.4.

4.4.1 Determining rheobase and chronaxie

1. Press to go to the start menu.
2. Select **Diagnostic programs**.
3. Select **Rheobase and chronaxie**.
4. If desired, change the **Polarity** and **Stimulation beep** settings.
5. Rotate intensity knob A to start the treatment. The set intensity is displayed in the screen.
6. Increase the intensity in steps of 0.1 mA , until you observe a tangible or visible contraction.
7. Press . The measured rheobase (in mA) is saved.
8. The equipment now doubles the rheobase (in mA). The pulse duration changes to 0.1 ms . Increase the pulse duration by Δ , until you observe a tangible or visible contraction.
9. Press . The chronaxie (in ms) is saved. The results screen appears.
10. If desired, press to save the data in the memory. See §4.5.1.

A	Rheobase	27
14:00	Pulse shape	Π
red+	Pulse time	1000 ms
0.0 mA CC	Pulse pause	1000 ms
	Polarity	red+
	Stimulation beep	ON
	Realise minimal muscle twitch	

A	Results(R+C)	
	Rheobase	0.3 mA
	Chronaxie	0.2 ms

4.4.2 Determining Rheobase and Accomodation Quotient (AQ)

1. Press \odot to go to the start menu.
2. Select **Diagnostic programs**.
3. Select **Rheobase and AQ**.
4. Determine the rheobase as with **Rheobase and chronaxie**. See §4.4.1.
5. Press \downarrow . The measured rheobase is saved.
6. The equipment now selects a triangular pulse. Increase the intensity in steps of 0.1 mA, until you observe a tangible or visible contraction.
7. Press \downarrow . The measured AQ is saved. The results screen appears.
8. If desired, press \odot to save the data in the memory. See §4.5.1.

4.4.3 I/T-curve

1. Select **Electrotherapy, Unidirectional, triangular pulse**.
2. Place the electrodes. Place the anode (+) on the spinal column (cervical for the upper limbs, dorsal for the rump or lumbal for the lower limbs). Place the cathode (-) on the motor point of the muscle to be investigated.
3. Set the pulse duration to 1000 ms.
4. Increase the pulse duration until you observe a tangible or visible contraction. Note this value in the graph. See §8.2.
5. Repeat steps 3 and 4 for the pulse durations 700 ms, 500 ms, 300 ms, 200 ms, 100 ms, 70 ms, 50 ms, 20 ms, 10 ms, 5 ms, 2 ms, 1 ms, 500 μ s, 200 μ s, 100 μ s.




4.4.4 Pain points



1. Press \odot to go to the start menu.
2. Select **Diagnostic programs**.
3. Select **Pain points**.
4. Select the diagnostic program for pain points.

4.5 Programs

You can save 20 of your own programs for later use: programs 500 up to and including 519. You can modify these programs for much-used or specific current shapes for a certain patient.



4.5.1 Saving a program

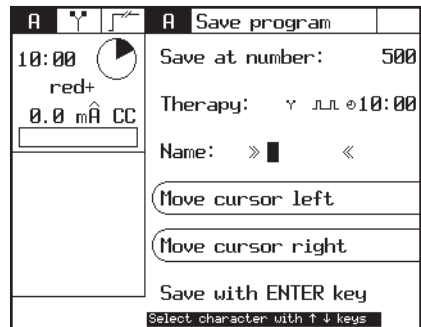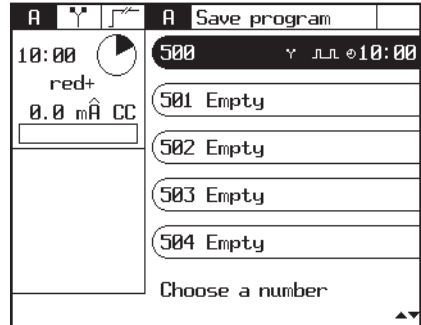
1. Select a therapy. See §4.1.
2. Change the settings for the patient. See §4.2.
3. Press .
4. Select **Save**.
5. Select a free program number with .
If desired, go to the following programs with Δ or ∇ .
6. Enter the name of the program. Use the name or the number of the patient, for example.
 - Select a character with Δ and ∇ .
 - Select **Cursor to left/right** to move the cursor.
7. Press  to save the program.





4.5.2 Selecting a saved program


Selecting a program by the name list

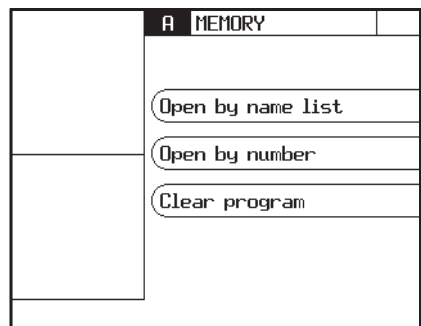1. Press .
2. Select **Open by name list**.
3. Go to the desired program with Δ or ∇ .
4. Select this program with .






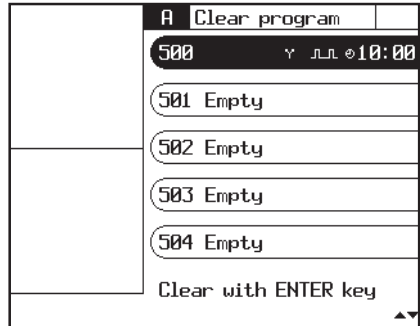
Selecting a program by the number

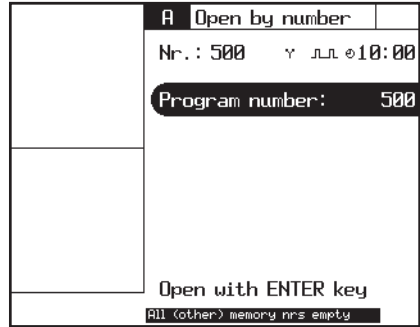1. Press .
2. Select **Open by number**.
3. Select the desired program with Δ or ∇ .
4. Press .






4.5.3 Clear a program


1. Press .
2. Select **Clear program**.
3. Select the program that must be deleted with .
If desired, go to the following programs with Δ or ∇ .
4. Press  to clear the program.



4.5.4 Editing a standard program

Standard programs have a program number that is lower than 50. You can only edit standard programs with the therapy key.

1. Select a program with the therapy key .
2. Press .
3. Select **Change default program**.
4. Press  to edit the standard program.

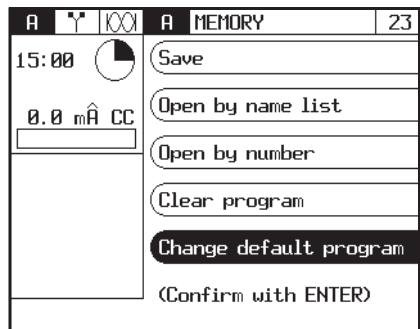


You can also save an edited standard program under a free program number. See §4.5.1.


You can reset the standard settings of the standard programs with **Reset Menu**. See §4.6.2.

4.6 System settings

With the system settings, you can adapt the Standard settings of the equipment. You cannot change the system settings during a therapy.

4.6.1 Changing the system settings

1. Press  for 5 seconds. The screen appears with the system settings.
2. Change the desired system setting.

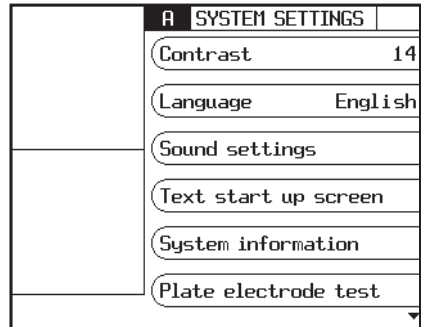

4.6.2 Parameters

Contrast (1 - 20)

The contrast of the display.

Language

The language selection: select the language with which the read-out must work.

Sound settings

Sound settings. See §4.6.3.

Text start up screen

The text that appears in the top of the start up screen, after the equipment is switched on. See §4.6.5.

Copy parameters (on, off)

Choose channel A and B the same or different is set by the copy parameter. See §4.2.2.

System information

System information of the equipment.

Always have this information available when you contact the technical service department.

Plate electrode test

Test the condition of the rubber electrodes. See §4.6.6.

Cable test

Test the cables. See §4.6.6.

Error history

The total number of error reports that the equipment has had and details about the last 10 error reports.

Always have this information available when you contact the technical service department.

Counter working hours (hours, minutes, sec.)


The time that the accessories for electrotherapy have been in use. For this, the output of the channel must have been higher than zero.

Reset menu

- **Reset working hours:** Set the number of working hours of a plate electrode to zero.
- **Reset programs 1-50:** This restores the standard settings of the standard programs. See §4.5.4.
- **Erase total memory:** Restores the standard settings of the standard programs and of the edited programs.

Press again to confirm.

4.6.3 Setting the sound

1. Press  for 5 seconds.
2. Select **Sound settings**.
3. Change the desired sound setting.

A	Sound settings	
	End of treatment	OFF
	Pressing a key	OFF
	ET Stimulation	OFF
	Beep volume	5
	ET bad contact	OFF

4.6.4 Parameters sound settings

End of treatment

On: A sound signal will be heard at the end of the treatment.

Pressing a key

On: A sound signal will be heard every time a key is pressed.

ET stimulation

On: A sound signal will be heard at each pulse of the electrotherapy.

Beep volume (min.1, standard 5, max.10)

The volume of the sound signals.


ET bad contact

On: A sound signal will be heard if the electrode does not make good contact with the skin.

4.6.5 Set text for start up screen

You can set your own text for the start up screen. For example, you can put your name or address information here.

1. Press \odot for 5 seconds, select **Text start up screen**.
2. Enter the name for the start up screen.
 - Select a character with Δ and ∇ .
 - Select **Cursor to left/right** to move the cursor.
3. Press \rightarrow to confirm the name.

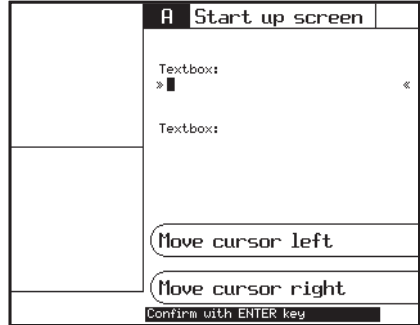

4.6.6 Cable test

1. Press \odot for 5 seconds. The **System settings** screen appears.
2. Select **Cable test**,
3. Connect the electrode cable to channel A with the electrodes.
4. Connect the test plug to the connectors of the cable.
5. Set the amplitude to 20 mA with rotary knob A.
6. If the cables function correctly, the following message will appear **Condition of cables: OK**.
7. Turn the amplitude back to 0 mA. Press \odot .

4.6.7 Rubber electrodes test

1. Remove the test plug and connect the electrodes to the electrode cable.
2. Place the electrodes on each other, without the sponges. Make sure that the electrodes make contact over the whole surface.
3. Set the amplitude to 20 mA with rotary knob A.
4. If the electrodes function correctly, the following message will appear **Condition of electrodes: OK**.
5. Turn the amplitude back to 0 mA.

5 INSPECTIONS AND MAINTENANCE

5.1 Inspections

Component	Check	Frequency
Electrode cables and electrodes	Damage Insulation intact	At least 1x per month
Equipment	Technical safety inspection. See §5.1.1.	At least 1x per year

5.1.1 Technical safety inspection

The 'Directive on Medical Devices' from the European Commission (93/42/EEG) requires that safe devices are used. It is recommended to perform a yearly technical safety inspection. If the legislation in your country or your insurer prescribes a shorter period, you must adhere to this shorter period.



- Only a technician authorised by GymnaUniphy N.V. may open the equipment or the accessories.
- The inspection may only be performed by a suitably qualified person. In some countries this means that the person must be accredited.

Inspection points

The technical safety inspection contains the following tests:

1. Test 1: General: Visual inspection and check on the operating functions
2. Test 2: Electrotherapy
3. Test 3: Electrical safety inspection: measurement of the earth leakage current and patient leakage current according to DIN/VDE 0751-1 ed. 2.0.

Inspection result

1. A registration must be maintained of the technical safety inspections. Use the inspection report in the appendix for this purpose. See §8.4.
2. Copy this appendix.
3. Complete the copied appendix.
4. Keep the inspection reports for at least 10 years.

The inspection is successful if all inspection items are passed.

Repair all faults on the equipment before the equipment is put back into operation.

By comparing the registered measurement values with previous measurements, a possible slowly-deteriorating deviation can be ascertained.

5.2 Maintenance

Component	Check	Frequency
Rubber electrodes	Cleaning. See §5.2.1.	After every treatment
EL sponges	Cleaning. See §5.2.2.	After every treatment
Fixation bandages	Cleaning. See §5.2.3.	If necessary
Vaginal, anal and rectal stimulation probe	Cleaning and disinfecting. See §5.2.4.	After each use



Accessories that come in contact with the body of the patient must be washed with pure water after the disinfection to prevent allergic reactions.

5.2.1 *Cleaning the electrodes*

1. Clean the electrodes in a non-aggressive soap solution or in a 70% alcohol solution.
2. Rinse the electrodes thoroughly with water.
3. Dry the electrodes.

5.2.2 *Cleaning the EL sponges*

1. Clean the EL sponges with a 70% alcohol solution.
 2. Rinse the EL sponges thoroughly with water.
- Or:
1. Soak the EL sponges with water.
 2. Put the EL sponges into boiling water for one minute.
 3. Soak the EL sponges with a saline solution to improve their conductivity.

5.2.3 *Cleaning the fixation bandages*

1. Clean the fixation bandages in a 70% alcohol solution or another disinfectant.
2. Rinse the fixation bandages in water.
3. Let the fixation straps dry.

5.2.4 ***Cleaning and disinfecting vaginal, anal and rectal stimulation probes***



- Considering the very personal and intimate character of these treatments, a probe may only be used for one patient.
- Never disinfect the probes in an autoclave. The probes can be damaged by the extreme temperature.

Immediately after every treatment

1. Clean the probe carefully with soap and water.
2. Place the probe in an HAC solution of 1% or in a 70% alcohol solution for at least 30 minutes.



- Read the instruction leaflet in the packaging of the HAC.
- Make sure that the probe connector does not get into the HAC solution.

3. Dry the probe with a clean towel.
4. Store the probe in a plastic bag that is provided with the name of the patient.

Before reusing the probe:

1. Clean the probe carefully with soap and water.
2. Apply an antiseptic lubricant to the probe. See §4.3.2.

6 MALFUNCTIONS, SERVICE AND GUARANTEE

6.1 Malfunctions

Component	Problem	Solution
Duo 200	Equipment cannot be switched on	See §6.1.1.
	Equipment does not react to commands or a fault report appears	See §6.1.3.
	Foreign language on the screen	Change the language. See §4.6.
EL sponges	Furring	Replace the sponges
	Bad conduction	Replace the sponges

6.1.1 *Equipment cannot be switched on*

1. Check if the mains voltage has failed.
2. Check if the main switch is switched on ("I").
3. Check if the power cord and the fuses are in order. If necessary, replace the fuse. See §6.1.2.
4. Contact your dealer if the equipment still cannot be switched on.

6.1.2 *Replacing a fuse*

1. Switch the main switch off ("O").
2. Unplug the power cord from the equipment.
3. Pull the fuse holder carefully out of the equipment. If necessary, use a screwdriver.
4. Replace the fuse. If necessary, order new fuses from your dealer.
5. Install the fuse holder and plug in the power cord.
6. Switch the main switch on again ("I").

6.1.3 *Equipment does not react to commands or an error message appears*

The safety system of the equipment has ascertained a fault. You cannot continue to work. An instruction usually appears on the screen.

1. Disconnect the connection to the patient.
2. Switch the main switch off ("O").
3. Wait 5 seconds and switch the main switch on again ("I").
4. Contact your dealer if the error message reappears.

6.2 Service



- Only a technician authorised by GymnaUniphy N.V. may open the equipment or the accessories to perform repairs. The equipment does not contain any components that may be replaced by the user.
- If possible, open the screen with the system settings before you contact the technical service department. See §4.6.

Service and guarantee are provided by your local GymnaUniphy dealer. The conditions of delivery of your local GymnaUniphy dealer apply.

If you have qualified technical personnel that are authorised by GymnaUniphy to perform repairs, your dealer can provide diagrams, spare parts lists, calibration instructions, spare parts and other information on request, for a fee.

6.3 Guarantee

GymnaUniphy and the local GymnaUniphy dealer declares itself to be solely responsible for the correct operation when:

- all repairs, modifications, extensions or adjustments are performed by authorised people;
- the electrical installation of the relevant area meets the applicable legal regulations;
- the equipment is only used by suitably qualified people, according to these user instructions;
- the equipment is used for the purpose for which it is designed;
- maintenance of the device is regularly performed in the way prescribed. See §5.;
- the technical life time of the equipment and the accessories is not exceeded;
- the legal regulations with regard to the use of the equipment have been observed.

The guarantee period for the equipment is 2 (two) years, beginning on the date of purchase. The date on the purchase invoice acts as proof. This guarantee covers all material and production faults. Consumables, such as sponges, adhesive electrodes and rubber electrodes, do not fall under this guarantee period.

This guarantee does not apply to the repair of defects that are caused:

- by incorrect use of the equipment,
- by an incorrect interpretation or not accurately following these user instructions,
- by carelessness or misuse,
- as a consequence of maintenance or repairs performed by people or organisations that are not authorised to do so by the manufacturer.

6.4 Technical life time

The expected life time of the equipment is 10 years, calculated from the date of manufacture. See the type plate for this information.

In so far as possible, GymnaUniphy will supply service, spare parts and accessories for a period of 10 years from the date of manufacture.

7 TECHNICAL INFORMATION

7.1 General

Dimensions Duo 200 (w x h x d)	266 x 275 x 100 mm
Weight Duo 200	3,650 kg
Weight including accessories	4,6 kg
Mains voltage	100 - 240 VAC, 50-60 Hz
Maximum power, in operation	85 VA
Safety class	Class I (earthed socket required)
Insulation	Type BF (floating patient circuit)
Fuses	2 x T2AL250V

7.2 Electrotherapy

7.2.1 General

Treatment time	0 - 60 min.
Current limitation	The smallest value: <ul style="list-style-type: none"> - 150% of the set value, or: - 110% of the maximum for the selected current shape
Accuracy	Set current value $\hat{m}A$ at 500Ω - typically $\pm 10\%$
CC/CV mode	For all current shapes, with the exception of medium frequency rectangular current
Polarity	Red-, red+ and alternating polarity, if applicable

7.2.2 Current shapes

Medium frequency rectangular current

Intensity 0 - 80 mA with 300 to 1000 Ω

Rectangular pulsed current, Triangular pulsed current, 2-5 Current (Ultra Reiz)

Pulse time 0,1 ms - 6 s

Pulse pause 1 ms - 6 s

Intensity of CC 0 - 80 mA with 300 to 1000 Ω

Intensity of CV 0 - 80 V_{pk} with I < 80 mA

MF, DF, CP, LP

Intensity of CC 0 - 80 mA with 300 to 1000 Ω

Intensity of CV 0 - 80 V_{pk} with I < 80 mA

ISO on / off

Conventional TENS, Low frequency TENS

Pulse time 10 - 650 μ s

Pulse shape symmetrical, asymmetrical

Frequency min. 1 - 150 Hz

Frequency max. 1 - 150 Hz

Intensity of CC 0 - 120 mA with 300 to 1000 Ω

Intensity of CV 0 - 120 V_{pk} with I < 120 mA

Random frequency TENS

See TENS currents, with the exception of:

Pulse frequency 1 - 150 Hz, with automatic stochastic frequency variation of +/-35% maximum

Burst TENS

See TENS currents, with the exception of:

Pulse frequency 20 - 150 Hz

Burst frequency 1 - 10 Hz

Rectangular surge current, Triangular surge current

Pulse time 0,1 - 5 ms

Pulse frequency 1 - 150 Hz

Intensity of CC 0 - 80 mA with 300 to 1000 Ω

Intensity of CV 0 - 80 V_{pk} with I < 80 mA

Biphasic surge current, Biphasic surge intrapulse interval(with a fixed interval between positive and negative pulses of 100 μ s)

Pulse time	10 - 650 μ s
Pulse frequency	1 - 150 Hz
Pulse shape	symmetrical, asymmetrical (only for Biphasic surge current)
Intensity of CC	0 - 120 mA with 300 to 1000 Ω
Intensity of CV	0 - 120 V _{pk} with I < 120 mA

2-pole medium frequency surge current, 4-pole interferential surge current

Carrier wave frequency	2 - 10 kHz
AM frequency	1 - 200 Hz
Intensity of CC	0 - 100 mA with 300 to 1000 Ω
Intensity of CV	0 - 100 V _{pk} with I < 100 mA

Expert parameters for NMES currents

Series duration (ON)	1 - 100 s
Series pause (OFF)	0 - 100 s

2-pole medium frequency current, 4-pole interferential current

Carrier wave frequency	2 - 10 kHz
AM frequency min.	0 - 200 Hz
AM frequency max.	0 - 400 Hz
Frequency variation mode	0/1/0, 1/5/1, 6/0/6, 12/0/12
Intensity of CC	0 - 100 mA with 300 to 1000 Ω
Intensity of CV	0 - 100 V _{pk} with I < 100 mA

4-pole interferential with rotating vector

See 2- en 4-pole interferential

Rotation time	0 - 20 s
Rotation angle	0 - 355°
Segment angle	0 - \pm 30°
Segment time	0 - 10 s

7.3 Environmental conditions








Temperature	+10 °C to +40 °C
Relative humidity	30% to 75%
Atmospheric pressure	700 hPa to 1060 hPa

7.4 Transport and storage

Transport weight	5,5 kg
Storage temperature	-20 °C to +60 °C
Relative humidity	10% to 100%, including condensation
Atmospheric pressure	200 hPa to 1060 hPa
Transport classification	Single piece, by post



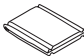

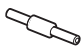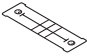
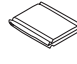








The transport and storage specifications apply to equipment in the original packaging.

7.5 Standard accessories

	Quantity	Description	Art. no.
	2	Two-conductor electrode cable	108.725
	2	Rubber electrode no. 2: 6 x 8 cm (per 2 pces)	109.959
	1	EL sponge no. 2 for electrode 6 x 8 cm (per 4 pces)	100.658
	4	Elastic fixation bandage - 5 x 60 cm	108.935
	1	Power cord ¹	100.689
	1	Test connector V/V - 4mm	108.919
	1	VAS score card	115.684
			NL: 117.268
			FR: 117.269
	1	User manual	EN: 117.271
			DE: 117.270
			ES: 327.382
	1	ET placing diagrams	117.276

¹ This power cord has a CEE 7/7 type plug. For countries with other outlets, a different power cord with the appropriate plug is supplied.

7.6 Optional accessories electrotherapy

	Quantity	Description	Art. no.
	1	Vaginal stimulation probe with 6-pole DIN plug	107.348
	1	Anal stimulation probe with 6-pole DIN plug	107.349
	1	Rectal stimulation probe	112.166
	2	Rubber electrode no. 1 - 4 x 6 cm	109.958
	2	Rubber electrode no. 3 - 8 x 12 cm	109.960
	4	EL sponge no. 1 for electrode 4 x 6 cm	100.657
	4	EL sponge no. 3 for electrode 8 x 12 cm	100.659
	4	Adhesive electrode, 3 cm diameter	326.799
	4	Adhesive electrode, 2,5 x 5 cm	326.810
	4	Adhesive electrode, 5 x 5 cm	326.821
	4	Adhesive electrode, 5 x 10 cm	326.832
	1	Adapter cable for adhesive electrode - 4 > 2 mm	113.334
	1	Pin electrode 15 mm diameter with grip and sponge	109.943
	10	EL sponges for pin electrode	109.944

Advice: Replace the electrode material at least every 6 months.

Article numbers can change in the course of time. Check the article numbers in the most recent catalogue or ask your dealer.

The drawings are merely indicative, no rights can be derived from them.


8 APPENDICES

8.1 Agents for iontophoresis

Agent	Property	Application and form
Calcium (+)	Analgeticum and sedative	Application: post-traumatic pain, distorsion, algodystrophic syndromes and neuralgia. Form: 2% calcium chloride solution.
Magnesium (+)	Analgeticum and fibrolyticum	Applications as with calcium. 10% magnesium chloride solution.
Iodine (-)	Sclerolyticum	Application: stubborn scars, cutaneous adhesences, sickness of Dupuytren, stiffness of joints and adhesive capsulitis. Form: 1-2% potassium iodine solution
Salicylate (-)	Anti-inflammation agent	Application: periphlebitis, osteoarthritis, ab-articular rheumatism, articular stiffness and adhesive capsulitis. Form: 2% sodium salicylate solution.
Procaine and lidocaine (+)	Anti-inflammation agent	Application: production of local anaesthesia, in the neuralgia of the trigeminal nerve, e.g. with acute inflammation. Form: 2% solution.
Histamine (+)	Revulsive and vasodilator	Application: degenerative and articular rheumatic pains, such as cramp. Maximum duration of iontophoresis: 3 min. Longer treatment causes allergic reactions and cephalgia. Form: 0,02% bicarbonate solution.
ColtramyI (+)	Myorelaxant	Application: contractures. Form: solutions up to 0,04%. 2 ml coltramyI (4mg/ ampoule), to be dissolved in 8 ml distilled water.
Indocid (-)	A.I.N.S.	Application: inflammatory illnesses. Form: 1% solution. 50 mg freeze-dried powder, to be dissolved in 5 ml distilled water.
Voltaren (-)	A.I.N.S.	Application: inflammatory illnesses. Form: 0,75% solution. 3 ml (75 mg/ampoule), to be dissolved in 7 ml distilled water.
Acetic acid	A.I.N.S.	Application: To dissolve deposition layers caused by ossifying myositis and periarticular ossification. Form: 2% water solution.

8.2 Diagnostic I/T-curve

Physiotherapist:	Date of investigation:
Name of patient:	Date of birth: M/F
Anamnesis:	
Evaluation (neuro-muscular):	Accommodation Quotient:
Rheobase: mA	Chronaxie: ms
Conclusion:	
Treatment:	

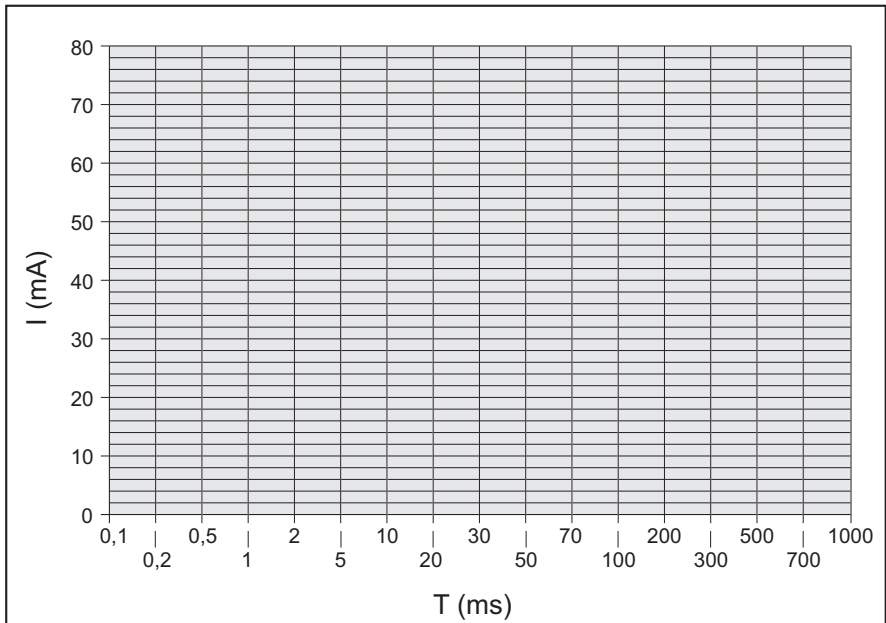

8.3 EMC directive

Use only cables, electrodes and US heads that are specified in this manual. See §7. The use of other accessories can have a negative effect on the electromagnetic compatibility of the equipment.


If you use the Duo 200 in the vicinity of other equipment, you must check that the Duo 200 is functioning normally.

The following paragraphs contain information about the EMC properties of the equipment.

8.3.1 Guidance and declarations

Guidance and manufacturer's declaration - electromagnetic emissions		
The 200-series devices are intended for use in the electromagnetic environment specified below. The customer or the user of a 200-series device should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The 200-series devices use RF energy only for their internal function. Therefore, their RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
	Class B	
Harmonic emissions IEC 61000-3-3	Class B	The 200-series devices are suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacturer's declaration - electromagnetic immunity			
The 200-series devices are intended for use in the electromagnetic environment specified below. The customer or the user of a 200-series device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact / ±8 kV air No loss of performance	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity must be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV power / ±1 kV I/O No loss of performance	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV diff. / ±2 kV comm. No loss of performance	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T (>95% dip in U_T) for 0,5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 sec	U_T - 100% (0,5 period) No loss of performance U_T - 60% (5 periods) No loss of performance U_T - 30% (25 periods) No loss of performance U_T - 100% (5 seconds) Device resets to a safe state. (60601-1 § 49.2)	Mains power quality should be that of a typical commercial or hospital environment. If the user of a 200-series device requires continued operation during power mains interruptions, it is recommended that the 200-series device be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	Not applicable	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE U_T is the a.c. mains voltage prior to application of the test level			

Guidance and manufacturer's declaration - electromagnetic immunity			
The 200-series devices are intended for use in the electromagnetic environment specified below. The customer or the user of a 200-series device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 V _{rms} AM 1 kHz 80% 150 kHz to 80 MHz	10 V.....0,15-80 Mhz 51 V.....6,78 Mhz 54 V.....13,56 Mhz 50 V.....27,12 Mhz 45 V.....40,68 Mhz	Portable and mobile RF communications equipment should be used no closer to any part of a 200-series device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 0,35\sqrt{P}$ $d = 0,07\sqrt{P}$ $d = 0,06\sqrt{P}$ $d = 0,07\sqrt{P}$ $d = 0,08\sqrt{P}$
Radiated RF IEC 61000-4-3	3 V/m AM 1 kHz 80% 80 MHz to 2,5 GHz	10 V/m...0,08-1,0 Ghz 26 V/m....1,4-2,0 Ghz 30 V/m...433,92 Mhz 30 V/m.....915 Mhz	$d = 0,35\sqrt{P}$ 80 MHz to 800 MHz $d = 0,70\sqrt{P}$ 800 MHz to 2,5 GHz $d = 0,12\sqrt{P}$ $d = 0,23\sqrt{P}$
Radiated RF ENV 50204	3 V/m CW 200 Hz d.c. 50% 895 MHz to 905 MHz	30 V/m.895-905 Mhz	$d = 0,23\sqrt{P}$
<p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b. Interference may occur in the vicinity of equipment marked with the following symbol:</p> 			
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 The guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			
<p>a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey can be considered. If the measured field strength in the location in which a 200-series device is used exceeds the applicable RF compliance level above, the 200-series devices should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the 200-series device. b Over the frequency range 150 kHz to 80 MHz, field strengths must be less than 10 V/m.</p>			

Recommended separation distances between portable and mobile RF communications equipment and the 200-series device

The 200-series device is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of a 200-series device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the 200-series devices as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 0,35\sqrt{P}$	80 MHz to 800 MHz $d = 0,35\sqrt{P}$	800 MHz to 2,5 GHz $d = 0,70\sqrt{P}$
0,01	0,04	0,04	0,07
0,1	0,11	0,11	0,22
1	0,35	0,35	0,70
10	1,11	1,11	2,21
100	3,50	3,50	7,00

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

8.4 Technical safety inspection

Duo 200 with serial number is / is not ¹ in good working order		
Location:	Inspection performed by: Name:	Owner: Name:
Date:	Initials:	Initials:

¹ Cross out what does not apply.

If a specific test does not apply to this equipment, place a mark in the NA (not applicable) column.

8.4.1 Test 1: General

	Yes	No	NA
1. The results of earlier safety inspections are available.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
2. The logbook is present.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
3. The type plate and the supplier's label are legible.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
4. The housing, adjusting knobs, keys and display are undamaged.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
5. The power connection and power cord are undamaged.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
6. The output connectors are undamaged.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
7. The electrode connectors and cables are undamaged.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
8. The automatic self-test at switch-on does not give an error message.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
9. The display does not show any defective points or lines.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

8.4.2 Test 2: Electrotherapy

	Yes	No
1. Connect loads of 500 Ω to both normal electrode pairs. Connect an oscilloscope to these pairs (black to ground).		
2. Select channel A, program 4: MF constant.		

	Yes	No
3. At maximum intensity, the output currents correspond within 10% with the values on the display.	<input type="checkbox"/>	<input type="checkbox"/>
4. The output signals correspond with figure 1.	<input type="checkbox"/>	<input type="checkbox"/>
5. The polarity changes to negative if "RED(-)" is selected.	<input type="checkbox"/>	<input type="checkbox"/>
6. The warning "Bad contact with the patient" is given if the load is disconnected.	<input type="checkbox"/>	<input type="checkbox"/>
7. Select channel B, program 4: MF constant. Select CC.		
8. At maximum intensity, the output currents correspond within 10% with the values on the display.	<input type="checkbox"/>	<input type="checkbox"/>
9. The output signals correspond with figure 1.	<input type="checkbox"/>	<input type="checkbox"/>
10. The polarity changes to negative if "RED(-)" is selected.	<input type="checkbox"/>	<input type="checkbox"/>
11. The warning "Bad contact with the patient" is given if the load is disconnected.	<input type="checkbox"/>	<input type="checkbox"/>
12. Remove the load, so that the unloaded output voltage can be measured.		
13. Select channel A, program 23: 2-pole medium frequency. Select CV.		
14. At maximum intensity, the output voltage corresponds within 10% with the values on the display.	<input type="checkbox"/>	<input type="checkbox"/>
15. The output signals correspond with figures 2 and 3.	<input type="checkbox"/>	<input type="checkbox"/>
16. The yellow lamp next to the output connectors lights if the intensity is not 0.	<input type="checkbox"/>	<input type="checkbox"/>
17. Select channel B, program 23: 2-pole medium frequency. Select CV		
18. At maximum intensity, the output voltage corresponds within 10% with the values on the display.	<input type="checkbox"/>	<input type="checkbox"/>
19. The output signals correspond with figures 2 and 3.	<input type="checkbox"/>	<input type="checkbox"/>
20. The yellow lamp next to the output connectors lights if the intensity is not 0.	<input type="checkbox"/>	<input type="checkbox"/>

Figure 1

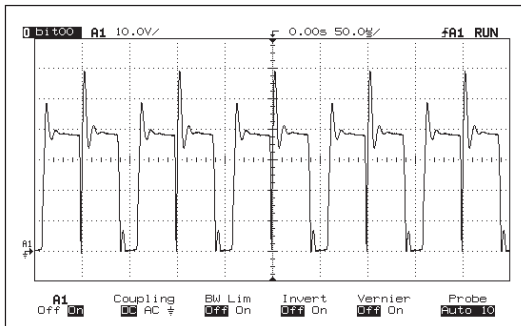


Figure 2


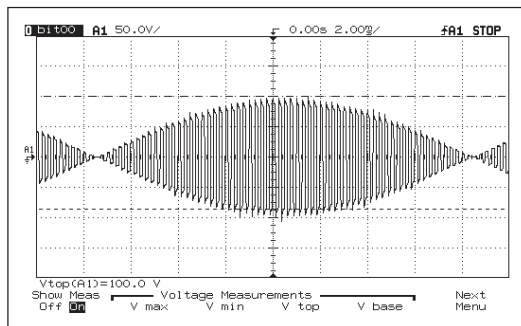


Figure 3

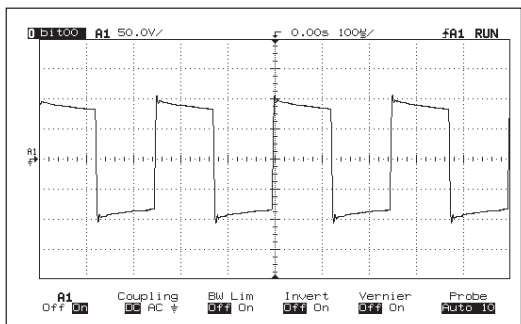

8.4.3 Test 3: Electrical safety test (VDE 0751-01)

	Yes	No
1. The resistance of the safety earth is less than 0.2 Ω	<input type="checkbox"/>	<input type="checkbox"/>
2. The housing leakage current is less than 1000 μA	<input type="checkbox"/>	<input type="checkbox"/>
3. The patient leakage current is less than 5000 μA	<input type="checkbox"/>	<input type="checkbox"/>

Notes:

8.5 Disposal

Take account of the following environmental aspects when disposing of the equipment and the accessories:

- The basic device, the cables and the electrodes fall under small chemical waste (electrical and electronic equipment waste). These components contain lead, tin, copper, iron, various other metals and various plastics, etc. Dispose according to national regulations.
- Sponges, sponge bags and gels contain only organic material and do not require any special processing.
- Packaging materials and manuals can be recycled. Deliver them to the appropriate collection points or include them with the normal household waste. This depends on the local organisation of the waste processing.

Notify your dealer about the disposal.

9 REFERENCE

9.1 Function overview

9.1.1 Therapy key

The numbers refer to the program numbers.

Electrotherapy

Unidirectional currents

Rectangular pulse.....	2
2-5 Current (UltraReiz).....	5
Triangular pulse	3
Medium freq. constant.....	4

Diadynamic currents

MF.....	18
DF.....	20
CP.....	21
LP.....	22

TENS currents

Conventional	6
Low frequency	7
Burst.....	10
Random frequency.....	9

9.1.2 System settings

Press  for 5 seconds

Contrast

Language

Sound settings

Text start up screen

Copy parameters

System information

NMES surge currents

Rectangular surge	11
Triangular surge	12
Biphasic surge	13
Intrapulse interval surge	14
2-pole MF surge	15
4-pole Interf. surge	17

Interferential currents

2-pole medium frequency	23
4-pole Interferential	26
4-pole Interf. vector	25

Plate electrode test

Cable test

Error history

Counter working hours

Reset menu

9.1.3 Objectives

The numbers refer to the program numbers.

Electrotherapy

Pain relief

Acute (VAS 75-100)	180
Subacute (VAS 50-75).....	181
Chronic (VAS 25-50)	182
Han Stim (VAS 40-90).....	195

Muscle detonising

Mild detonisation.....	183
Normal detonisation	184

Muscular training

Prevention atrophy	229
Atrophy	
Clear atrophy.....	227
Slight atrophy.....	228
Muscle function, specific	
Endurance.....	230
Strength	237
Explosive strength	239
Resistance	250
Pelvic re-education	
Stress incont. level 1	54
Stress incont. level 2	55
Urge incontinence.....	56
Mixed incontinence	57
Denervation	
Triangular pulse.....	60
Rectangular pulse	61

9.1.4 Indication list

ET: Electrotherapy

The numbers refer to the program numbers.

Acrocyanosis, ET

Intensive, local.....	80
Mild, segmental.....	135
Specific points	81

Arteriosclerosis, ET

Intensive, local.....	191
Mild, segmental.....	135
Specific points	81

Arthralgia, ET

Local	180
Local + regional	84
Specific points	85

Arthrosis, ET

Local	80
Local + regional	84
Specific points	181

Atrophy, ET

Slight atrophy.....	187
Prevention of atrophy	146

Brachialgia, ET

Acute	180
Subacute.....	181
Chronic.....	182

Burger, ET

Intensive, local.....	80
Mild, segmental	135
Specific points	81

Bursitis, ET

Acute	180
Subacute.....	97

Cellulitis, ET

Activ. subcut. muscle.....	116
Breakdown fat tissue.....	117

Cervicobrachialgia, ET		Low back pain, ET	
Acute	180	Acute	180
Subacute.....	181	Subacute	120
Chronic	182	Chronic.....	137
Cervicoceph. syndr., ET		Lumbalgia, ET	
Subacute.....	180	Acute	180
Chronic	181	Subacute	120
Contractures, ET		Chronic.....	137
Subacute.....	84	Myofasc. trigger point, ET	
Chronic	118	Subacute.....	97
Coxarthrosis, ET		Chronic.....	182
Subacute.....	84	Neuralgia, ET	
Chronic	181	Acute	180
Decubitus, ET		Subacute	102
With infection	52	Chronic.....	181
Without infection	51	Oedema, ET	119
Dysmenorrhoea, ET		Over activity bladder, ET.....	196
Acute	84	Phantom pain, ET	
Subacute.....	181	Acute	180
Epicondylitis, ET		Chronic.....	181
Local	139	Post surgical pain, ET	
Local + regional	97	Acute	180
Fractures, ET	53	Subacute	181
Frozen shoulder, ET		Posttraum. dystrophy, ET	
Subacute.....	84	Acute	180
Chronic	118	Subacute	120
Gonarthrosis, ET		Chronic.....	181
Subacute.....	84	Raynaud, ET	
Chronic	137	Intensive, local	80
Herpes Zoster, ET		Mild, segmental	135
Acute	180	Specific points	81
Subacute.....	181	Sciatica, ET	
Hypertonic muscles, ET		Acute	180
Subacute.....	120	Subacute	102
Chronic	97	Chronic.....	181
		Spasticity, ET	
		Detonisation.....	136
		Reciprocal inhibition.....	131

Sprain, ET
 Subacute..... 84
 Chronic 80

Südeck's dystrophy, ET
 Acute, segmental 180
 Subacute, regional 120
 Chronic, central..... 181

Tendinitis, ET
 Local 139
 Regional 97

Ulcus Cruris, ET
 With infection 52
 Without infection 51

9.1.5 Diagnostics

Rheobase and chronaxie 27
 Rheobase + AQ..... 28

Pain points
 Electrotherapy
 Superficial Nerve points . 109
 Superficial motor points . 110
 Deep motor points 129
 Painful area/zone 111

9.1.6 Contra indication

Electrotherapy

General

- High fever
- Severe cardiovascular problems
- Psychological problems
- Cancer with tumor metastasis
- Generalised tuberculosis

Specific absolute

- On demand pacemakers

Specific relative for monophasic pulses

- Skin lesions
- Skin infections
- Thrombosis, thrombophlebitis
- Varices
- Increased risk to haemorrhage
- Superficially implanted materials
- Heart disease, rhythm disorder
- Decreased sensibility
- Locat. near sinus caroticus
- Menses
- Pregnancy

Specific for relative biphasic pulses

- Skin infections
- Thrombosis, thrombophlebitis
- Heart disease, rhythm disorder
- Decreased sensibility
- Locat. near sinus caroticus
- Pregnancy

9.2 Literature

A literature list can be sent on request. Please contact GymnaUniphy.

9.3 Terminology

absolute muscle power: The maximum total tension that a muscle can produce.

accomodation: The ability of the nerve tissue to protect itself against stimulations that slowly increase in strength.

Pulse time	Delay in action potential of rectangular pulse: triangular pulse	Accomodation Quotient (AQ)
500 ms	1:1.5 to 1:3	1,5 - 4
1000 ms	1:2 to 1:6	2 - 6

active trigger point: A point that, with stimulation (pressure, stretch or electrical pulse), besides the local pain also generates a projected pain in the area that the patient is complaining about.

antalgic: The pain is reducing.

atrophy: Deterioration in the nourishment state of organs. As a result, the organs become smaller or shrink.

chronaxie: The time threshold that is required for a muscle contraction or a sensory impression, after the occurrence of the necessary minimum required stimulation.

denervation: Switching-off or weakening of the innervation (paralysis).

durability: Being able to frequently repeat a muscle contraction.

epithelisation: Recovery of the epithelium over the bottom of the wound. A unidirectional current can stimulate the epithelisation. Epithelisation can also be activated by an external electrical stimulation.

explosive muscle power: The highest tension that a muscle can produce in the shortest possible time.

hyperalgesia: An increased sensitiveness for pain. Apply a modified dosage in the case of acute hyperalgesia.

injury current: A small unidirectional current between the epidermis and the corium, which occurs after a wound. This current activates the recovery process. With a slow recovery process, an external unidirectional current can be applied to realise the same effect.

innervation: The effect of the nerves on the working of the muscles or glands.

iontophoresis: The flow of ions through a tissue by means of a galvanic current.

isometric contraction: A muscle contraction whereby the length of the muscle remains constant. The external resistance of the muscle must be at least as large as the power that is generated by the contraction. Under isometric circumstances, especially the tension in the muscle increases and muscle cramp is avoided.

loadability: The (maximum) load that can be carried.

loss of muscle tone: The state of tension of muscles reduces.

Myofascial Trigger Point (MTP): A trigger point that is located in the myofascial tissue. The MTP is located in a hard cord of a muscle. The MTPs can be localised with **Pain points** in the **Diagnostics program**.

Neuro Muscular Electro Stimulation (NMES): Contraction of an innervated muscle or muscle group by means of low or medium frequency electrostimulation. The purpose of NMES is to improve or maintain the movement.

pain threshold: The lowest level of stimulation that causes pain.

pain tolerance threshold: The level of stimulation that can just be tolerated by the patient. The pain tolerance threshold is past the pain threshold.

re-innervation: The restoration of the innervation.

responsiveness: The degree to which a tissue or organ reacts to a stimulation. With a high responsiveness, a mild treatment is desired. With a low responsiveness, a more intensive treatment can be desired. Make a good estimate of the responsiveness to determine the correct dosage.

rheobase: The minimum galvanic current strength required with the stimulation of the nerve to cause a muscle contraction.

sclerolysis: The solution of a hardening of the tissue. The tissue can be chemically and electrically softened with a cathode in combination with chlorine or iodine.

skin etching: Electro-chemical reactions that can be threatening for tissues and organs, especially for the skin. With correct application, a desired effect occurs, for example improvement of the circulation. Skin etching occurs with current shapes that have a direct current component.

slow twitch muscle fibre: Muscle fibres with a low contraction speed. The fibres are fairly thin, produce a small amount of power and have a low fatigue level. See also type I muscle tissue.

tetanic contraction: A persistent muscle contraction, on the basis of several contraction waves that are simultaneously in a muscle. You can cause tetanic contractions with an NMES surge current.

tone: The tension state of tissues.

trophic: The state of nourishment.

type I muscle tissue: Muscle tissue with a low contraction speed.

type II muscle tissue: Muscle tissue with a high contraction speed. Set the parameters as follows for stimulation with NMES:

NMES parameter	type I	type II
Pulse time	Long	Short
Pulse frequency	Low	High
Pulse amplitude	-	High
Series duration and series pause	Short	Long
Treatment time	Long	-

VAS score: Score on the Visual Analogue Scale (VAS). Tool for evaluating a clinical complaint from the patient. This usually concerns the degree to which pain is felt. With a high VAS score, a mild treatment is usually adequate. With a lower VAS score, a more intensive treatment is desired.

10 INDEX

A

Abbreviations 4
 Absolute muscle power 67
 Accessories 50
 optional electrotherapy 51
 standard 50
 Accomodation 67
 Accomodation Quotient 32
 Active rest 28
 Active trigger point 67
 Agents for iontophoresis 53
 Anal stimulation probe
 cleaning 41
 perform electrotherapy 25
 Antalgic 67
 Atrophy 67

B

Burst 28

C

Cable test 35, 37
 Carrier wave 28
 Channel
 set 22
 Chronaxie 31, 67
 Cleaning 40
 Connection 11
 Constant Current 28
 Constant Voltage 28
 Contra indication 66
 selection 21
 Contrast 11, 35
 Copy parameters 35
 Counter working hours 36
 Current shape groups 16
 Current shapes 16, 17

D

Denervation 67

Diadynamic currents 17, 63
 Diagnostic 66
 perform 31
 program selection 20
 Display 15
 symbols 16
 Disposal 62
 Durability 67

E

Editing a standard program 34
 EL sponges
 cleaning 40
 Electrical safety 8
 Electrode
 cleaning 40
 test 37
 Electrode placing 29
 Electrotherapy 23
 direct selection 19
 optional accessories 51
 parameters 27
 perform 23
 read-out values 27
 safety 9
 sequential steps 25
 technical information 47
 EMC 8
 EMC directive 55
 Environmental conditions 49
 Epithelisation 67
 Error history 36
 Explosive muscle power 67

F

Fixation bandage
 cleaning 40
 Frequency 29
 Function overview 63
 Functional test 11

G

Guarantee 44

H

Hyperalgesia 68

I

I/T-curve 32, 54

Indication list 20, 64

Injury current 68

Innervation 68

Inspections 39

Installation 11

Intensity screen 23

Interferential currents 18, 63

Interruption 23

Iontophoresis 27, 68

Isodynamic 29

Isometric contraction 68

L

Language 11, 35

Liability 9

Loadability 68

Loss of muscle tone 68

M

Maintenance 40

Malfunctions 43

Medical Devices Directive 9

Muscle detonising 64

Muscle tissue

type I and type II 69

Muscular training 64

Myofascial Trigger Point 68

N

Name list 33

NMES 68

currents 18

surge currents 63

O

Objectives 19, 64

Off time 29

On time 29

On2 amplitude 29

On2 frequency 29

P

Pain points 32

Pain relief 64

Pain threshold 68

Pain tolerance threshold 68

Parameter symbols

electrotherapy 17

Placing 11

Placing rubber electrodes 24

Placing the adhesive electrodes 24

Plate electrode test 35

Polarity 29

Polarity change 29

Prevention of explosion 8

Program 33

clear 34

number selection 20

saving 33

selecting 33

Pulse pause 29

Pulse shape 29

Pulse time 29

Purpose 7

R

Rectal stimulation probe

cleaning 41

perform electrotherapy 25

Re-innervation 68

Replacing a fuse 43

Reselling 12

Reset menu 36

Responsiveness 69

Rest amplitude 29

Rest frequency 30

Rheobase 31, 32, 69

Rotation angle 30

Rotation mode 30

Rotation time 30

S

Safety 7

instructions 7

technical inspection 39, 59

Sclerolysis 69

Segment angle 30

Segment time 30

Selection

therapy 19

Seq. step nr. 26, 30

Seq. steps 30

treatment 26

Service 44

Skin etching 69

Slow twitch muscle fibrel 69

Sound

set 36

Sound settings 35

Step time 30

Stimulation beep 30

Stop 23

Stop treatment 23

Storage 12

conditions 50

Sweep mode 31

System information 35

System setting

parameters 35

System settings 35

changing 35

T

Technical information 47

Technical life time 45

TENS currents 18, 63

Terminology 67

Tetanic contraction 69

Text start up screen 35

set 37

Therapy

direct selection 19

program selection 20

selection via indication list 20

selection via objectives 19

set 21

start 21

Therapy keys 63

Tone 69

Transport 12

conditions 50

Treatment

interruption 23

sequential steps 26

stop 23

Treatment time 31

Trophic 69

U

Unidirectional currents 17, 63

V

Vaginal stimulation probe

cleaning 41

perform electrotherapy 25

VAS score 70

© 2004, GymnaUniphy N.V.

All rights reserved. Nothing from this publication may be copied, stored in an automated data file, or made public, in any form or in any way, be it electronically, mechanically, by photocopying, recordings or in any other way, without prior written permission from GymnaUniphy N.V.