
Radarmed 950+ Therapy guide Protocols

Behandeltabellen
Treatment tables
Behandlungstabelle
Tableau de traitement
Tabla de tratamiento

Copyright: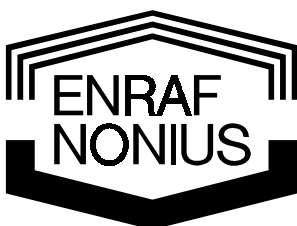

**Enraf
Nonius**

Enraf-Nonius B.V.
P.O. Box 12080
3004 GB ROTTERDAM
The Netherlands
Tel.: +31(0)10 - 20 30 600
Fax: +31(0)10 - 20 30 699

Order number : 1435.770-41
December 2005

Radarmed 950+ Therapy guide Protocols

Behandeltabellen
Treatment tables
Behandlungstabelle
Tableau de traitement
Tabla de tratamiento

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1. Microwave therapy with the Radarmed	15
1.1. Introduction	15
1.2. Continuous and pulsed microwave therapy	15
1.3. Sandbags according to KIRN	15
2. Description of the radiators	15
2.1. The radiators	15
2.2. Distance of the radiator to the skin	15
3. Rules for microwave treatment	16
3.1. Preparation of the patient	16
4. Dosage	16
4.1. Introduction	16
4.2. Levels subjective sensitivity to heat	16
4.3. Rules for dosage calculation:	17
5. Contra-indications	17
5.1. Absolute contra-indications	17
5.2. Relative contra-indications	17
5.3. Special groups of patients	17
5.4. Note	17
6. Microwave treatment Tables - Indications ...	18
6.1. The Protocol memory of the Radarmed 950+	18
6.2. Explanation of the footnotes and abbreviations	18
6.3. Microwave Treatment table Dermatology	19
6.4. Microwave Treatment table Gynecology	20
6.5. Microwave Treatment table Internal medicine	21
6.6. Microwave Treatment table neurology	22
6.7. Microwave Treatment table Surgery and Orthopedics	22

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1. MICROWAVE THERAPY WITH THE RADARMED

1.1. Introduction

The therapeutic effect of warmth has been put to use ever since the earliest beginning of medical science. Today the therapist is able, with the aid of modern technology, to direct warmth endogenously to the precise area where it is needed.

Radarmed microwave units operates at a frequency of 2450 MHz, corresponding to a wavelength of 12 cm. At this wavelength, the high-frequency energy is absorbed particularly well in every kind of tissue containing a high percentage of water.

Microwaves penetrate the subcutaneous fatty tissue almost without loss, being converted to warmth only when they reach the musculature or organs well supplied with blood, as well as in the skin, of course.

Apart from the inherent safety of the method itself, modern Radarmed units, thanks to microcomputer technology, attain an extremely high standard of safety in use, to the benefit of both patients and operating personnel.

1.2. Continuous and pulsed microwave therapy

The possibilities of microwave therapy is significantly widened by the use of the pulsed mode. In this mode, microwave pulses with a fixed, high intensity are generated. In contrast with continuous microwave the higher intensity of the pulses briefly warms up the deeper lying tissues. At the same time, the same dosage rules can be used as for treatment with continuous microwave, since the sensation of warmth felt by the patient is the same with either form of treatment. Pulsed microwave can be used in cases where only a low dosage is permissible, but nevertheless deeper lying regions have to be reached.

1.3. Sandbags according to KIRN

The effect of microwave therapy can be improved by combining it with the use of sandbags according to KIRN. When microwaves are radiated through a layer of dry natural sand of specific thickness, the dielectrical adaptation of the field to the organic tissue is improved.

The interposed medium sand is comparable with the lens in the field of optics. The improved concentration of the microwaves reduces losses through diffusion. The mechanical adaptation of the sandbag to the form of the body makes local targeting of the microwave field possible. This is to enable specific areas which are unduly sensitive to heat to be excluded from the radiated field.

The areas covered by the sandbags the field is homogeneously concentrated, more intensive and capable of deeper penetration. Microwave application can thus be additionally modified by means of the microwave/sand method.

In addition to the properties of microwave therapy as mentioned above, mention must be made of the simplicity of use thanks to the unipolar application. With 3 different radiators, it is able to deal easily with all the applications required in general and clinical practice.

2. DESCRIPTION OF THE RADIATORS

2.1. The radiators

The Radarmed unit accessories include 3 microwave radiators to cover all the applications encountered in medical practice.

Local field radiator

The local field radiator enables greater depth effects to be obtained over smaller areas. It is the most universal and therefore most used radiator.

Longitudinal radiator

The longitudinal radiator is designed for the treatment of the extremities.

Large field radiator

The large field radiator is used for treating large areas of the body. The form of the radiator is adapted to the contours of the body. This radiator is very suitable for treating isolated areas such as the shoulder or knee joint, as well as larger muscle groups in the back and shoulder regions.

2.2. Distance of the radiator to the skin

The following distances of the radiator to the body surface is necessary*.

Distance from body surface:

Rondveldstraler	Approx. 10 cm.
Langveldstraler	Approx. 5 cm.
Grootveldstraler	Approx. 1 cm.

* The essential distance for the radiators is determined by the internal construction..

3. RULES FOR MICROWAVE TREATMENT

3.1. Preparation of the patient

- Before the start of treatment, the patient should remove all articles of clothing containing metal (lurex) and metal items in pockets or any other such articles which are liable to come within the radiation zone. Such articles include rings, chains, clasps, bracelets, jewellery, watches, piercings, articles with metal fittings such as bras, suspender belts, corsets, hearing aids and metallic and electronic prostheses, etc.
- Moisture concentrations heat up more intensively in the treatment zone of the radiator. In leather or synthetic clothing occurs easily increased perspiration. It is therefore recommended to uncloth the bodyparts which are to be treated and, in the case of strongly perspiring, dried off (folds in the skin). Dry bandages can be irradiated quite safely. Diathermy should not be applied in regions covered by moist bandages or adhesive tape.
- Parts of the body containing metallic endoprotheses such as medullary pegs, metal splinters, metallic joints, pins, spirals, skull plates, dental fillings and crowns, wires and so on, should not be subjected to treatments or only at extremely low dosage levels.
- When treating narrow parts of the body (e.g. a wrist) it should be ensured that the radiator is set up so that sensitive parts of the body (e.g. eyes, testicles) are not in the irradiated area. In these cases it is recommended to use a smaller radiator.
- It should be made possible for the patient to adopt a relaxed and calm posture during the entire treatment.

4. DOSAGE

4.1. Introduction

As long as the rules of treatment are adherent to, no harm will be incurred by patients as the result of microwave radiation. As the blood circulation increases strongly in the region the generated warmth is quickly dissipated and there is no built-up of heat. Injuries can be sustained only as the result of gross errors of treatment (heavy overdose), exactly as with any other form of physical or medical therapy.

If goes without saying that markedly aqueous tissue which is poorly vascularized, such as the eyes and testicles, should be treated only at very low dosage levels.

4.2. Levels subjective sensitivity to heat

In practice 4 levels of sensitivity to heat can be distinguished.

1 Imperceptible heat:

Skin heated to just below the heat sensitivity threshold. Set dosage so that warmth is just barely perceptible, then reduce it by a few scale divisions.

2 Slide heating:

Just perceptible warmth.

3 Comfortably warmth:

Pleasant, easily tolerable feeling of warmth.

4 Hot:

Just tolerable, almost burning feeling of warmth.

The decisive factor for all dosage calculations must be the patient's own sensitivity to heat. During the first treatment of a series, the patient should be asked immediately about his subjective feeling.

- Always set the dosage according to the individual patient and never in accordance with a fixed scheme.
- Every patient has a different susceptibility to warmth and this generally reaches a steady state only after about 5 minutes.
- The patient's sensitivity to warmth may vary during the course of a treatment (adaptation).
- The patient must be asked to report an increase of the warmth sensation. The dosage must then be reduced.
- The dosage must not be increased under any circumstances as a result of any lessening of the feeling of warmth during treatment.

- When sensibility disorders is suspected the subjective warmth perception of the patient is checked by means of the skin temperature. In order to avoid overheating in such cases, the dosage must be adjusted with extreme care. Apply only small doses for short periods to the treatment zone.

Microwave therapy should never be applied to parts of the body under local anaesthetic because this might lead to local burns.

4.3. Rules for dosage calculation:

The **more acute** a disorder, the milder the dosage should be (1-2); apply frequently with short treatment times (3-10 minutes) daily, if necessary twice daily, applications over a limited period (6-10 sessions).

The **more chronic** the disorder (or its resulting conditions), the higher the dosage (2-3) should be; apply less frequently with long treatment times (15-20 minutes); 2-3 x weekly over several months (up to 18 or more sessions).

5. CONTRA-INDICATIONS

5.1. Absolute contra-indications

- Patients with implanted heart pacemakers.
- Metallic endoprotheses in the treatment region.
- Malignant tumors. An increase in temperature may result in an increase in the rate of growth of the tumor or metastatic secondaries.
- Active tuberculosis or other active specific disorder.

5.2. Relative contra-indications

- Acute infectious disorders, especially when accompanied by high fever.
- Acute inflammatory processes of the bones and joints. Warmth may, in certain circumstances, induce or accelerate necrosis of the tissue. Furthermore, local heating may be more severe than anticipated in areas where the blood supply is reduced or inhibited as a result of lacking normal heat dissipation by blood circulation. Particular care is necessary when deciding the dosage if diathermy is used in the presence of ischemia or severe tissue damage.

- Traumata and vascular lesions for the first 24 to 36 hours after the incident, in order to prevent the danger of internal and external bleeding.
- Disorders in sensitivity to heat.
- Anaesthetized areas.
- In the region of the abdomen during menstruation.
- Patient with intra-uterine pessaries.
- Osteoporeses.
- Increased tendency towards bleeding and use of anitcoagulants.
- Thrombophilia.
- The epiphysial disks must also be exempted with children until the cessation of growth.
- The application of diathermy in the region of the eyes. When treating the head e.g. nose or frontal sinuses, it is recommended to take into account the reduced vascularization of the eyes and provide a pair of protective goggles in order to avoid heat stress. Microwave treatment of the eyes should be carried out by an eye specialist.
- Wearers of contact lenses. Contact lenses may inhibit heat dissipation and thus cause overheating of the eyes. They should be removed before treatment in the head region.
- Treatment in the region of the testicles. Special protective measures must be taken here since the testicles are extremely sensitive to heat, which may cause permanent damage or even sterility.
- Pregnancy. Special care must be taken to protect the embryo or foetus during microwave treatment in the region of the abdomen.

5.3. Special groups of patients

- Infants are best undressed completely before treatment. Their physical volume calls for special care in calculating the dosage and repeated checking of the skin temperature by hand.
- Never leave infants or sensitive adults, especially elderly patients, unattended during treatment.

5.4. Note

This list makes no claim to completeness. For the application of microwave therapy we refer to chapter 6 of this booklet. In this you will find information on the radiators, their applications, the recommended dosage and the number of treatments. The statements are related to the in paragraph 2.2 mentioned distances of the radiator to the skin and in respect of contra-indications and other treatment criteria from case to case.

6.1. The Protocol memory of the Radarmed 950+

The Radarmed 950+ is delivered with an empty memory and should be filled by the user. For this purpose two different options are available:

1. Own treatment suggestions are programmed into the memory (see operating manual Radarmed 950+, paragraph 8.3.).
2. Pre-programmed treatment suggestions are loaded into the memory.

Loading of pre-programmed treatment suggestions

The treatment suggestions of paragraph 6.3 upto 6.7 which are marked with a program number are latently present in the internal memory of the Radarmed 950+. These protocols can be loaded into the protocol memory at choice by following the next procedure:

Loading memory

- Switch off the unit in case it was turned on.
- Press the "Program retrieve key", "Program store key" and "Program key protocols" while the unit is turned on and the display shows "Configuring protocols".
- After a few seconds a sound signal is audible, the protocol memory is now loaded.
- The Radarmed is ready for operation.

6.2. Explanation of the footnotes and abbreviations

In the treatment tables the hereunder mentioned footnotes and abbreviations are used.

Footnotes:

- 1 = Treat florid processes under drug protection (antibiotica, sulphonamides), especially if infectious exacerbation may be anticipated in the treatment zone.
- 2 = It is essential to clarify the diagnosis precisely prior to microwave therapy ! Exclude specific disorders because of the danger of tissue liquefaction.
- 3 = If possible, combine microwave therapy with simultaneous aerosol inhalation.
- 4 = Apply detumescent prior to microwave treatment
- 5 = It is recommended to administer 2 tablets of an antiphlogistic (Tanderil, Voltaren or similar) 45 min. prior to microwave treatment.
- 6 = Effect is potentised by combination with drug therapy, which should be timed to obtain the highest blood level at the time of microwave treatment.
- 7 = Effect is potentised by combination with spasmolytica applied in good time intravenously or rectally.
- 8 = No microwave treatment for 24-36 hours after traumata with vascular lesions, because of the danger of secondary hemorrhage.

Radiator type:

- R = local field radiator
L = longitudinal radiator
M = large field radiator (Mulden radiator)

Remarks:

- P = pulsed microwave have been proven to be effective
Zz = the use of sand bags is advantageous
Zb = the use of sand bath is advantageous

6.3. Microwave Treatment table Dermatology

P. No.	Indication			Intensity stage	Setting [W]	Duration [min.]	Frequency of treatment	No. of sessions	Remarks
A1	Abscess	1	R	1 - 2	20 - 30	5 - 10	Daily to every 2 days	5	
	Alopecia (in preparation for scalp massage)		R	2 - 3	30 - 70	10	as necessary		
A2	Carbuncle	1	R	2 - 3	30 - 60	10	every 2 days	2 - 3 weeks	
	Cosmetic Application (alopecia, itch, dandruff)		R	2 - 3	30 - 60	10 - 15	Daily	6	P
A3	Frostbite, local, fresh		R	2	20 - 50	5 - 10	Daily	6	P
A4	Frostbite, local, old		R	3	40 - 80	10	Twice weekly	10 early begin	
A5	Furuncle	1	R	2	30 - 60	10	Daily	5	
	Gonorrheal arthritis, old		R	3 - 4	60 - 120	15 - 20	Twice weekly	10	
	Gonorrheal arthritis, chronic	1, 2	R	3 - 4	60 - 120	20	Twice weekly	10	
A6	Herpes zoster		M L	3	50 - 80	10	every 2 days	10	
	Paronychia, initial stage	1	R	1 - 2	20 - 40	5 - 10	every 2 days	5	
A7	Post-herpetic neuralgia		M	3	50 - 100	15 - 20	Daily to every 2 days	10	P
	Radiation injury, low grade (e.g. sunburn, X-ray erythema)		R M	2 - 3 2 - 3	30 - 70 40 - 80	10 - 15 10 - 15	Daily to every 2 days	5 - 10 5 - 10	
	Resorption acceleration percutaneous		R M	2 - 3 2 - 3	30 - 60 40 - 80	10 - 15 10 - 15	Daily Daily	6 - 10 6 - 10	
	Stab-cell pyoderma, small (blisters, pustules, impetigo, acne, sycosis)	1, 2	R	2	30 - 50	5 - 10	Daily	5	
	Sudoriparous abscess	1	R	1 - 2	20 - 40	5 - 10	Daily	10	P
	Warmth application		M L	3 - 4	50 - 150	30 and longer	Once only		
	Whitlow, initial stage	1	R	1 - 2	10 - 30	5 - 10	every 2 days	5	
	X-ray ulcer		R	2	25 - 50	10 - 15	every 2 days	10	

6.4. Microwave Treatment table Gynecology

P. No.	Indication			Intensity stage	Setting [W]	Duration [min.]	Frequency of treatment	No. of sessions	Remarks
B1	Adnexitis, unilateral	1, 2, 6	R	2	25 - 50	10	Daily to every 2 days	5 - 10	P
	Adnexitis, bilateral		M L	2	30 - 60	10	Daily to every 2 days	5 - 10	P
	Adnexitis, chronic	1, 2	R	3 - 4	60 - 120	15	Twice weekly every 2 days	12 and more	
			M L	3 - 4	60 - 140	15			
	Amenorrhea (esp. secondary)		M L	3 - 4	60 - 120	10	Daily	6	
	Bartholinitis		R	2 - 3	30 - 70	10	Daily	up to 10	
	Douglas infiltrate (+ stump infiltrate)	2, 6	M	3	40 - 90	10 - 15	every 2 days	5 - 10	
	Dysmenorrhea		M L	3	50 - 100	10 - 15	Daily	3 - 5	
B2	Low back pain		R	3 - 4	60 - 120	20	every 2 days to twice a week	5 - 10 - 15	
			M L	3 - 4	70 - 140	20		5 - 10 - 15	
B3	Mamma abscess	1	R	2 - 3	30 - 60	5 - 10	every 2 days	5 - 10	P
B4	Mastitis	1	R	2 - 3	30 - 60	10	every 2 days	5 - 10	P
B5	Mastodynia (premenstr.) Galactostasis (puerpera)	2	R	2	25 - 50	15	Daily to every 2 days	5 - 10	P
	Menstrual irregularity		L	3	25 - 50	15	Daily	6	
			M	3	30 - 60	15	Daily	6	
	Metritis	2, 6	R	2 - 3	30 - 70	10 - 15	Daily	5 - 10	ZzP
	Metritis (endo-, myo-, peri-subacute)		M	2 - 3	40 - 80	10 - 15	Daily	5 - 10	ZzP
	Metritis, chronic	2, 6	R	3	40 - 90	10	Daily to twice weekly	5 - 10	
	Metritis, parametritis		M	3	50 - 110	10	Twice weekly	5 - 10	
	Ovarian cysts		M	2	30 - 60	15	Daily	6	P
	Parametropathia spastica (+ spastic pelveopathy)		R	3 - 4	60 - 120	15	Twice weekly to every 2 days	5 - 10	ZzP
			M	3 - 4	60 - 140	15		5 - 10	ZzP
	Salpingitis catarrh.	2, 6	R	3 - 4	60 - 120	10	every 2 days	5 - 10	
			M	3 - 4	60 - 130	10	every 2 days	5 - 10	

6.5. Microwave Treatment table Internal medicine

P. No.	Indication			Intensity stage	Setting [W]	Duration [min.]	Frequency of treatment	No. of sessions	Remarks
C1	Bronchial asthma	3	M	3	50 - 100	15	every 2 days	15	P
			R	3	40 - 90	15	Twice weekly	15	P
C2	Bronchiectasis	3	M	3	50 - 100	15	Twice weekly	15	
			R	3	40 - 90	15	Twice weekly	15	
C3	Bronchitis chronica	3, 6	M	3	50 - 100	15	every 2 days	15	
			R	3	40 - 90	15	to twice weekly	15	
C4	Cholecystitis	1	R	3	40 - 90	10	every 2 days	5 - 10	
	Circulatory disorders of the acra (fingers, toes, lower leg, acrocyanosis, endarteritis, dead finger, thrombophlebitis in consolidation stage)		L	3	40 - 100	10 - 15	Twice weekly	12 and more	Zz
			R	3	40 - 90	10 - 15	Twice weekly		Zz
	Oesophageal spasms	2, 7	R	3	40 - 90	10	Twice weekly	10	
C5	Pleuritis chronica sicca	2	M	3	50 - 100	15	Twice weekly	10	
			R	3	40 - 90	15	Twice weekly	10	
C6	Pleuritis chronica residual condition	2	M	3 - 4	80 - 120	15	Twice weekly	10	
			R	3 - 4	60 - 120	15	Twice weekly	10	
C7	Raynaud's disease		L	3	40 - 100	10 - 15	Twice weekly	12 and more	Zz
			R	3	40 - 90	10 - 15	Twice weekly		Zz
	Renal colic (nephropathia spastica)	7	M;L	2 - 3	30 - 60	10	Daily	5	P
			L	2 - 3	25 - 50	10	Daily	5	P
	Spastic cholecystopathy	7	R	2 - 3	30 - 60	10	Daily	5	P
	Spastic constipation	2, 7	M	3	50 - 100	15	Daily	up to 5	
			R	3	40 - 90	15	Daily	up to 5	
	Tracheitis	3	M	3	50 - 100	15	every 2 days	10	
			R	3	40 - 80	15	every 2 days	10	
	Vascular neurosis (angioneurotic spasms)		L	3	40 - 100	10 - 15	Twice weekly	12 and more	Zz
			R	3	40 - 90	10 - 15	Twice weekly		Zz
	Venous thrombosis follow-up treatment	1, 2	L	2 - 3	25 - 50	10	Twice weekly	10	P

6.6. Microwave Treatment table neurology

P. No.	Indication			Intensity stage	Setting [W]	Duration [min.]	Frequency of treatment	No. of sessions	Remarks
D1	Intercostal neuralgia	5	M, L	3	50 - 100	10	Daily to every 2 days	5 - 10	P
D2	Ischialgia		M, L	3	50 - 100	10	Daily to every 2 days	5 - 10	P
D3	Neuralgia in general		R M	3 3	40 - 90 50 - 100	10 15	Daily to every 2 days	5 - 10 5 - 10	P P
D4	Neuritis	6	R	2 - 3	30 - 70	10	every 2 days	10	P
	Occipital neuralgia	5	R	3	40 - 90	10	Daily to every 2 days	5 - 10	P
	Root pain (radiculitis)	5	L	3	50 - 100	15	every 2 days	12 and more	Zz
	Sciatica, genuine, acute	2, 5	L	2 - 3	40 - 80	15	Twice weekly	12 and more	
	Sciatica, genuine, chronic		L	3 - 4	70 - 130	15	Twice weekly	12 and more	

6.7. Microwave Treatment table Surgery and Orthopedics

P. No.	Indication			Intensity stage	Setting [W]	Duration [min.]	Frequency of treatment	No. of sessions	Remarks
E1	Arthritis (except acute specific arthritis)	1, 2	R	2	25 - 50	10 - 15	every 2 days	5 - 10	P
E2	Arthropathy, acute	2, 5	R	2 - 3	40 - 70	10 - 15	Daily	5 - 10	P
E3	Arthropathy, chronic		R	3 - 4	60 - 120	15 - 20	Twice weekly	12 and more	
E4	Arthropathy, functional	5	L	3	60 - 100	15 - 20	Daily	5 - 10	
	Arthropathy, muscular side-effects, periosteum and tendon insertion irritation		R	3	50 - 100	15 - 20	Daily	5 - 10	
E5	Arthrosis deformans, small joints (Herberden's nodes)		R	3	50 - 90	15 - 20	Twice weekly	12	
E6	Arthrosis deformans, medium joints (e.g. genu = Gonarthrosis)		R M	4 4	60 - 110 80 - 120	20 20	Twice weekly Twice weekly	12 and more	
E7	Arthrosis deformans, large joints (e.g. Coxarthrosis)		R M	4 4	70 - 140 80 - 150	20 20	Twice weekly Twice weekly	12 and more	Zz Zz
E8	Articular rheumatism, chronic	6	R	3 - 4	40 - 120	15 - 20	Twice weekly	12 and more	
E9	Bechterew's disease (symptomatic only)		M L	3	60 - 100	15 - 20	Weekly	extending for months	
	Brachialgia	2;5	L R	3 3	50 - 100 40 - 90	10 - 15 10 - 15	Daily to every 2 days	10 10	

P. No.	Indication			Intensity stage	Setting [W]	Duration [min.]	Frequency of treatment	No. of sessions	Remarks
E20	Bursitis	5	R	2	25 - 50	20	every 2 days	12	P
	Calf cramp		M L	3 - 4	70 - 120	10	as necessary		Zz
	Cervicobrachial syndrome	5	L R	3 3	50 - 100 40 - 90	15 15	Daily to every 2 days	5 - 10 5 - 10	
	Coccygodynia		L R	3 3	50 - 100 40 - 90	10 - 20 10 - 20	Daily to every 2 days	5 - 10 5 - 10	
E22	Contusions (limbs, joints, trunk)	5, 8	R M	2 - 3 2 - 3	30 - 70 40 - 80	15 15	Daily to every 2 days	5 - 10 5 - 10	
	Coxarthrosis		R M	4 4	70 - 140 80 - 150	20 20	Twice weekly Twice weekly	12 and more	Zz Zz
	Coxitis (except specific)	1, 2	R M	3 3	40 - 90 50 - 100	15 15	every 2 days every 2 days	12 and more	
	Discogenic syndrome (without X-ray manifestation; acute prolapse, for repositioning, loosening of myogelosis, ischiatic complaints, radiculitis, paresthesia)	5	M L	3	50 - 100	15	every 2 days to twice weekly	5 - 10	Zb
E23	Distorsions	5, 8	R	2 - 3	30 - 70	15	Daily	5 - 10	P
	Dropsy (e.g. genu and other joints)	2	R	2 - 3	30 - 70	15	every 2 days	12	
E16	Epicondylitis	2, 5	R	3	40 - 90	10 - 15	Daily	5 - 10	Zz
	Extension of spinal column (supportive)		M L	3	60 - 100	15	every 2 days	5	
E24	Hematoma (not before 24 h) also in conjunction with distorsions, luxations, subperiosteal H.	5, 8	R	2 - 3	30 - 70	15	Daily	5 - 10	P
E14	Ischialgia, acute	5	M L	2 - 3	40 - 80	15	Daily	5	
E15	Ischialgia, chronic		M L	3 - 4	60 - 120	20	Twice weekly	12	
E25	Lumbago/lumbalgia, acute	5	M L	3	50 - 100	15	every 2 days to twice weekly	1 - 5 (10)	Zb
E26	Lumbago/lumbalgia, chronic		M L	4	80 - 140	15	Twice weekly	12 and more	Zb
	LVC syndrome, chronic	2	M L	3 - 4	60 - 130	15	every 2 days to twice weekly	10 - 15	Zb
	Meniscopathy	5	R M	2 - 3 2 - 3	30 - 70 40 - 80	15 15	every 2 days to twice weekly	12 and more	P P

P. No.	Indication			Intensity stage	Setting [W]	Duration [min.]	Frequency of treatment	No. of sessions	Remarks
E21	Muscle cramps (e.g. SC myogelosis, calf cramp)	5	M L	3 - 4	70 - 120	10 - 15	as necessary		Zz
	Muscular rheumatism, acute	5	R	2 - 3	30 - 70	10	every 2 days	10 - 15	P
	Muscular rheumatism, chronic		R	4	70 - 120	20	Twice weekly	10 - 15	P
E19	Myalgia in general ("Myositis")	5	R M	3 3	40 - 90 40 - 100	10 - 20 10 - 20	Daily Daily	1 - 5 - 10 1 - 5 - 10	
	Myogelosis	5	R M	3 3	40 - 90 40 - 100	10 - 15 10 - 15	Daily Daily	1 - 5 - 10 1 - 5 - 10	
	Myositis (genuine inflammation)	5	R M L	2 - 3 2 - 3	30 - 70 40 - 80	15 15	every 2 days every 2 days	5 - 10 5 - 10	P P
	Osteochondrosis, LVC, CVC		M L	3 - 4	60 - 120	15	Twice weekly	12 and more	Zz
	Paravertebral myogelosis	5	M L	3	60 - 100	15	Daily to every 2 days	5 - 10	Zz
E10	Periarthritis humeroscapularis, chronic		R M	3 - 4 3 - 4	60 - 120 80 - 140	15 - 20 15 - 20	every 2 days to twice weekly	10 - 15 10 - 15	Zz Zz
E11	Periostitis	5, 8	R	2	25 - 50	5 - 10	Daily	10	P
	Phantom-limb pain (post-operative)		R	2 - 3	60 - 90	10	Twice weekly	for months	P
E12	Polyarthritis rheumatica	6	R M	3 - 4 3 - 4	40 - 120 50 - 120	15 15	twice weekly twice weekly	12 and more	Zz Zz
	Preparation for most Physiotherapeutic treatment forms (connective tissue massage, chiropractice, extension, massage etc.)		M L	3 - 4	60 - 120	10 - 20	as necessary		Zz
	Residual conditions (after tearing of a ligament, distorsions, hematoma, contusions, luxations, etc.)	5, 8	R M L	2 - 3 2 - 3	30 - 70 40 - 80	15 15	Daily Daily	5 - 10 5 - 10	P P
	Spondylarthrosis deformans		M L	4	80 - 150	15	Twice weekly	12 and more	Zz
E13	Spondylosis		M L	3 - 4	50 - 130	15	Twice weekly	12	Zz
	Static dysfunction of the locomotor system		M L	3	60 - 110	15	every 2 days	12	
	Styloiditis	5	R	2 - 3	30 - 70	10 - 15	Daily	5 - 10	Zz
E17	Tendinitis	5	R L	3 3	40 - 90 60 - 100	15 - 20 15 - 20	every 2 days every 2 days	5 - 10 5 - 10	Zz Zz
E18	Tendovaginitis	5	L R	2 - 3 2 - 3	40 - 80 30 - 70	15 15	every 2 days to twice weekly	10 10	Zz Zz
	Thoracic syndrom SC, chronic		M L	3 - 4	60 - 130	15	Twice weekly	10 - 15	
	Torticollis		R	3	40 - 80	10	Daily	4 - 5	

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